



Instruction for Use

Radiation Shield Apron

Type / Composition

Lead (TRU-COMFI) and Lead-free (TRU-LITE) apron are composed with a waterproof outer cover. There are four types of apron : Front type, Front-back type, Vest-skirt type, and X-type shield apron. The size is as follows : S, M, L, LL

Purpose of use and expected effects

1. Provides protection for the patient, operator or the assistant from radiation exposure.

Method of use

1. Wear it and fix it with the Velcro strap or buckle.

Precautions

1. Avoid water or humidity as much as possible.
2. Do not sit on the apron or place things on it.
3. Do not use dry cleaning, laundry or autoclave.
4. Keep away from the edges of machines or sharp tools.
5. Handle with caution. Do not throw the apron or hit it with great force because it may cause deformation or breakage.
6. In case of disposal, dispose the product in accordance with the guidelines laid down by the local authorities concerning the disposal of industrial waste.

Storage and usage period

1. Keep away from direct sunlight, strong ultraviolet rays, high temperature, high humidity, and dusty places.
2. When storing the apron in use, always keep the apron hanging on the dedicated hanger for the X-ray protective apron storage without bending.
3. When packaging the product, the cushioning material is inserted between the products to prevent self-compression. If it is opened during storage or unpacked and repackaged, it is important to keep it folded well like the first packing (The cushioning material is inserted between them) to keep it from crumpling.

Maintenance and inspection

A. Daily monitoring

1. Make sure there is no visible damage.
2. Confirm that there is no breakage of the inner barrier with your hands.
3. Use alcohol and soapy solution (neutral detergent) for cleansing.

B. Periodic inspection

Aprons should be X-rayed periodically to check for cracks in the X-ray protective lining.

Manufacturer
CWORLD