



## THREE WAYS STOP COCK WITH EXTENSION TUBE

Size/Model : 10cm,25cm,50cm,75cm,100cm

### DIRECTION FOR USE:

- Peel open the pouch.
- Unscrew the protective cap from female luer.
- Connect male portion of device with female luer of tubing
- Hold three way stop cook upright and open clamp of device and three way stop cook allow the solution to displace air entirely from tubing.
- Close clamp.
- Connect Three way stop cook venipuncture device.
- Re-use of this device may change its mechanical of biological features and may cause device failure, allergic reactions or infections.
- Should not be used for the treatment of children, pregnant women or nursing mothers.

### CAUTION

- Do not use if package is open or damaged
- Do not store at extreme temperature and humidity.
- Store in cool and dry place.
- Discard after single use.



Suzhou Yudu Medical Co., Ltd

Address: Eastern Xinzhuang (Zhangqiao) Town, 215552 Changshu, China



GIÁM ĐỐC

*Nguyễn Cải Triền*