



# HYSOKAT

Hysterosonography - Hystero-salpingo-foam-sonographie (HyFoSy)

## PRODUCT DESCRIPTION


- HYSOKAT features specially developed to perform Hystero-salpingo-foam sonography or traditional hysterosonography:
  - Central pre-curved soft catheter displaying:
    - Centimetric graduations,
    - Tactile marks on the grip indicating the direction of the curve.
  - External sliding tube providing:
    - A **very soft and atraumatic** cone at the distal end, for an effective and harmless occlusion of the cervix,
    - A jointly fixed ring, at the proximal end for precise positioning.

## INDICATIONS

- Hysterosonography - Hystero-salpingo-foam-sonographie (HyFoSy).

## RECOMMENDATIONS FOR USE

- Gently insert HYSOKAT through the cervix into the uterus.
- Push the external tube forward until an effective occlusion is obtained.
- Slowly inject the selected echogenic preparation.
- Upon completion of the procedure, gently withdraw HYSOKAT and properly discard according to clinical waste treatment procedures.

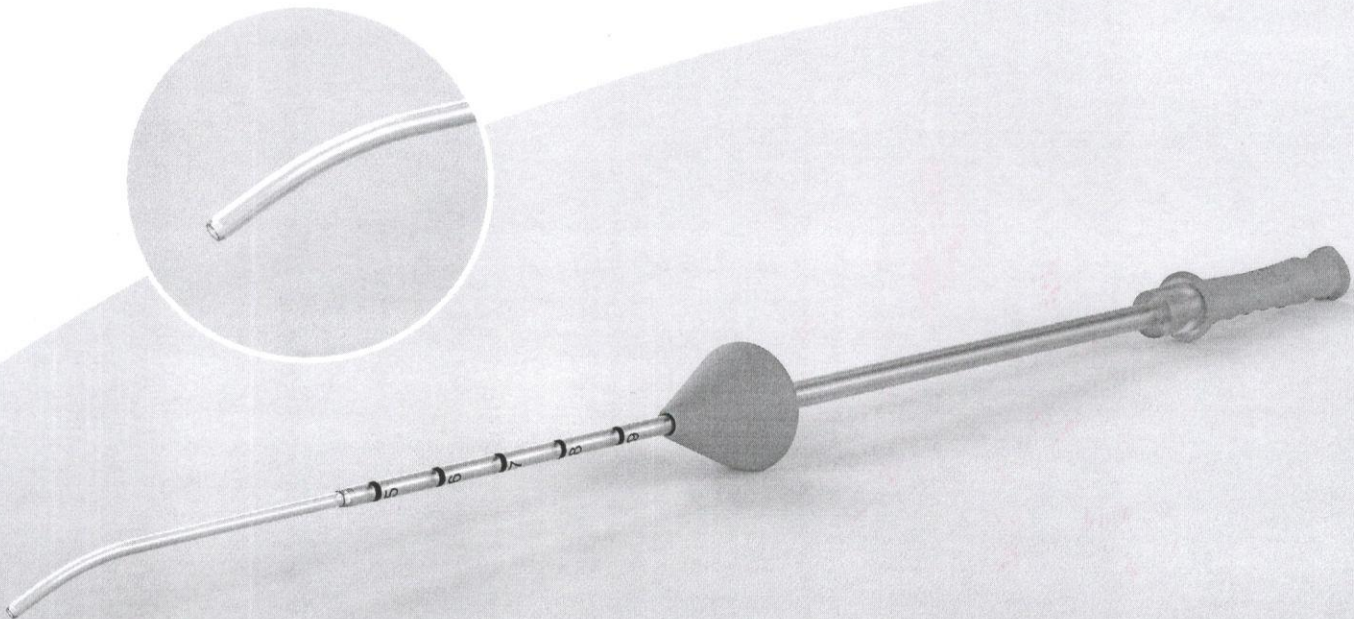

| References | Product | Sales Unit                   | Length  | Sterilisation  |
|------------|---------|------------------------------|---------|----------------|
| EBT01700   | HYSOKAT | 25 units individually packed | 24.5 cm | Ethylene oxide |

/AN/10