


Shiva Products
 H.O. : 214, Bluebell Industrial Estate, Chinchpada, Vasai East, Palghar - 401 208, Maharashtra, India.
T-Fill (Instruction For Use)
 (Temporary Filling Material)

Product Description of T-FILL

T-Fill is a ready to use temporary filling material (Type II- Temporary Cement for bases and temporary restoration) in paste form, self-curing under humidity, to be used for temporary filling of cavities during restorative or endodontic treatment of teeth.

Ingredients:

Zinc Oxide, Barium Sulphate, Zinc Sulphate, Calcium Sulphate, Talc, Vinyl Acetate, Acetate of Ethylene Glycol, Coloring & Flavoring agent

Flavor : Mint

Indication of use:

- 1) For use as temporary filling material during endodontic and restorative treatment of teeth.
- 2) For use as Temporary Inlay material

Properties

- 1) Eugenol free offers non irritating properties.
- 2) Higher adhesion to teeth structure.
- 3) Micronised and homogeneously mixed for grain free consistency. Gives best adaptability.
- 4) Quick setting in moist conditions.
- 5) There is minor but even expansion on setting so no cracks develop, giving best sealing Hence protecting to Pulp.

Applications

Use a cement carrying instrument to fill the required quantity of T-Fill into the prepared cavity after air drying. A ball burnisher can be used to adapt the T-Fill to the margins of the cavity for better marginal seal. The hardening process starts after few minutes of contact with moisture or patient's saliva. Avoid any exposure to chewing pressure for about 2 hours after application. Pressure free application is recommended when used in conjunction with a root canal dressing material. Inform patient ,In case of fracture of the restoration, get restoration repeated as soon as possible.

Setting time

Initial setting time : From 9 minutes, once it comes in contact with saliva.
Final setting time : 24 hours, once it comes in contact with saliva.

Contraindications

- 1) Do not use the product if the patient is allergic to any of the ingredients.
- 2) Not Intended for Permanent cementation.

Product Handling:

Product being moisture sensitive, close jar immediately after dispensing the material, do not use wet Instrument to dispense material. Moisture contamination would change consistency of product and reduce its usability. Product to be used by qualified dental professional or qualified dental assistant.

Storage

After opening store in refrigerator (Do not freeze)

Packing

40gm pack

Disposal

Dispose of contents/container in accordance with the local/regional/national/international regulations & as per MSDS

Shiva Products












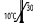

Mfg. site: B/18, Dwarka Ind. Estate, Naik Pada, Valiv,
Vasai (E), Dist. Palghar - 401 208, Thane, Maharashtra. INDIA.
Website: www.maarcidental.com
Email ID: care@shivaproducts.net
Customer Care: +919967806543


Obelis S.A
Boulevard General Wahis 53
1030 Brussels, Belgium
 Tel: +(32) 2 732 59 54, Fax: +(32) 2 732 60 03
 E-mail: mail@obelis.net


 0123

| | |
|----------|------------|
| Revision | 05 |
| Date | 29-07-2020 |

These instructions for use should be kept for the duration of product use.

-  Symbol for "BATCH CODE"
  Symbol for "USE BY"
  Symbol for "NON STERILE"
  Symbol for "KEEP DRY"
-  Symbol for "CAUTION"
  Symbol for "CONSULT INSTRUCTION FOR USE"
  Symbol for "CATALOGUE NUMBER"
-  Symbol for "EUROPEAN COMMUNITY AUTHORIZED REPRESENTATIVE"
  Symbol for "DO NOT USE IF PACKAGE IS DAMAGE"
-  Symbol for "KEEP OUT OF THE REACH OF CHILDREN"
  Symbol for "MANUFACTURER NAME & ADDRESS"
-  Symbol for "TEMPERATURE LIMITATION"
  Symbol for "KEEP AWAY FROM SUNLIGHT"

The material has been developed solely for professional dental use. Application should be carried out strictly according to the instruction for use. Liability cannot be accepted for damages resulting from failure to observe the instructions of the stipulated area of applications. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in this instruction sheet. Description and data constitute no warranty of attributes and are not binding.