

HYALOSAN vaginal gel with hyaluronic acid



Medical device – Instructions for use



Read instructions for use carefully before you start using **HYALOSAN vaginal gel**.
Keep it, you may need to read it again.

10 tubes with applicator

The volume of one tube: 7.5 ml

HYALOSAN vaginal gel with hyaluronic acid maintains naturally moist and slightly acidic environment of the vaginal mucosa thanks to the content of lactic acid.

It is used to support regeneration and restoration of the natural state of the vaginal mucosa in the case of problems caused by:

- vaginal dryness associated with hormonal changes in all stages of a woman's life, especially during menopause, after labour, during breast-feeding, after chemotherapy and radiotherapy or gynaecological operations,
- vaginal atrophy manifested by itching, burning, redness or unpleasant feelings during intercourse.

It can be used as a lubricant gel in the case of insufficient moisture during intercourse.

The gel is colourless and odourless.

Ingredients:

Sodium Hyaluronate, Lactic Acid, Glycerin, Hydroxyethylcellulose, Sodium Benzoate, Aqua.

Hyaluronic acid is important, natural component of many tissues of human body including the skin and mucous membranes. Together with vaginal mucus it forms a protective gel, which:

- instantly protects and moisturizes mucous membrane,
- supports natural mechanisms of vaginal mucosa moisturization,
- accelerates natural process of its regeneration.

Hyaluronic acid also prevents atrophy of vaginal mucosa and supports its restoration in a period of natural or chemotherapy- or surgically-induced menopause.

In physiological conditions, vaginal epithelium (upper layer of vaginal mucosa) is colonized by bacteria, which produce lactic acid. **Lactic acid** ensures acidic environment of vagina, which backwardly supports the growth of natural bacterial microflora and also creates unfavourable conditions for the occurrence of pathogenic bacteria.

Instructions for use:

Wash your hands thoroughly and check the integrity of protective foil before using. Take the tube out of the protective foil and remove the cap of vaginal applicator. Apply while lying on your back. Insert the applicator into the vagina as deeply as possible and press the tube to push out the entire content. Remove the applicator while constantly pressing the tube to prevent the gel getting back into the tube. Stay lying on your back a few minutes after application so the gel can evenly cover the vaginal mucosa. Wash your hands again.

Dosage:

One tube daily for the first ten days, then every second to third day for at least 5 to 6 weeks.

As a prevention of problems, Hyalosan vaginal gel with hyaluronic acid is also suitable for long-term use with dosage 1 to 2 tubes per week.

Apply preferably before the bedtime or before planned sexual intercourse.

Use a panty liner as a protection against mild gel discharge from vagina.

HYALOSAN vaginal gel is intended for vaginal use only.

Consult your doctor in case of hypersensitivity to any of the gel ingredients, during pregnancy and when concomitantly using other vaginal preparations (including vaginal contraception).



Do not use during menstruation or in case of increased vaginal secretion.

Do not use if the packaging is damaged.

Do not use after the expiry date listed on the package.

Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer or the national competent authority.

Keep out of reach of children.

Store at temperatures from +5 to +25 °C.

Read the instructions	
Batch number	
Expiry date	
Do not re-use	
Store at a temperature from +5 ° C to +25 ° C	
Do not pollute the environment - dispose of used packaging in appropriate container	
Manufacturer	



Dr. Müller Pharma s.r.o.

U Mostku 182, 503 41 Hradec Králové, Czech Republic

info@muller-pharma.cz, tel.: +420 495 809 111

www.muller-pharma.com

Distributor:

Date of last revision: