



HYALOSAN vaginal suppositories

Sodium hyaluronate 5 mg in one suppository

Dr. Müller
PHARMA

Medical device - Instructions for use



Read instructions for use carefully before you start using **HYALOSAN vaginal suppositories**. Keep it, you may need to read it again.

HYALOSAN vaginal suppositories with hyaluronic acid maintain naturally moist environment of the vaginal mucosa.

They are used to support regeneration and restoration of the natural state of the vaginal mucosa in the case of problems caused by:

- vaginal dryness associated with hormonal changes in all stages of a woman's life, especially during menopause, after labour, during breast-feeding, after chemotherapy and radiotherapy or gynaecological operations,
- vaginal atrophy manifested by itching, burning, redness or unpleasant feelings during intercourse.

Ingredients:

Sodium Hyaluronate, Tocopherol, Centella Asiatica, Malva Sylvestris, Chamomilla Recutita, Melaleuca Alternifolia, Silica, Hydrogenated C12-18 Triglycerides.

Hyaluronic acid is important, natural ingredient of many tissues of human body including the skin and mucous membranes.

Together with vaginal mucus it:

- instantly protects and moisturizes mucous membrane,
- supports natural mechanisms of vaginal mucosa moisturization,
- accelerates natural process of its regeneration.

Hyaluronic acid also prevents atrophy of vaginal mucosa and supports its restoration in a period of natural or chemotherapy- or surgically-induced menopause.

Instructions for use:

Wash your hands thoroughly before use. Remove the suppository from the aluminium foil by tearing over the tip of the suppository and insert it into the vagina as deep as possible. Apply while lying on your back. Stay lying on your back a few minutes after application, after insertion into vagina, vaginal suppository softens due to body temperature and evenly covers the entire surface of the vaginal mucosa. Pollution of underwear occurs during application, use a panty liner for protection if needed. Apply preferable before the bedtime.

Dosage:

One suppository daily for first ten days or as recommended by your doctor.

For the treatment of vaginal dryness caused by menopause, apply one suppository for first 10 – 20 days, followed by a long-term treatment with one vaginal suppository 2 times per week.

HYALOSAN vaginal suppositories are intended for vaginal use only.

Consult your doctor in case of hypersensitivity to any of the suppository ingredients, during pregnancy and when concomitantly using other vaginal preparations (including vaginal contraception).

Do not use during menstruation or in case of increased vaginal secretion.

Do not use if the packaging is damaged.

Do not use after the expiry date listed on the package.



Any serious adverse event that has occurred in relation to the use of device should be reported to the manufacturer or the national competent authority.

Keep out of reach of children.
Store at temperature from +5 to +25 °C.



Read the instructions	
Batch number	
Expiry date	
Store at a temperature from +5 ° C to +25 ° C	
Do not pollute the environment - dispose of used packaging in appropriate container	
Manufacturer	
Date of manufacture	



Dr. Müller Pharma s.r.o.,

U Mostku 182, 503 41 Hradec Králové, Czech Republic
tel.: +420 495 809 111, info@muller-pharma.cz

www.muller-pharma.com

Distributor:

Date of last revision: