

RubyFlow

Light Cure Flowable Composite



Product

RubyFlow is flowable light cure radio opaque composite.

Composition

Methacrylate polymers (44%)
Inorganic fillers (56%)
Filler particle size 0.04-0.70 micron
The fillers contain barium glass, mixed oxides, photo-initiators and stabilizer.

Indications

- Anterior restorations (Class III, IV)
- Very small posterior restorations (Class I, II)
- Class V restorations (Cervical caries, root erosion, wedge-shaped defects)
- Veneering of discoloured anterior teeth
- Preventive resin restorations in molars and premolars
- Repair of composite and ceramic veneers
- As the first layer of Class I and II restorations

Shade Selections

Select the desired shade with Vita shade guide in daylight conditions. For better shade determination, clean the teeth. The shade should be selected with the tooth still moist.

Application

Apply the composite once etching and bonding is completed. Place increments of no more than 2mm into the cavity. Use a non-metallic instrument.

Transparent matrix strips may be used. Place cap on syringe immediately after use.
Light cure each layer with a halogen or LED light.
Curing Time:

40 sec. for lighter shades (e.g. A1, A2, A3)

60 sec. for lighter shades (e.g. A3.5, B3)

Finishing:

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Inter-proximal finishing is accomplished by fine grit finishing strips.

NOTE:

- Place cap on syringe immediately after use!
- Do not use any resin to adjust viscosity of composite material.

-Neither store the composite material in proximity of eugenol containing products, nor let the composite come into contact with materials containing eugenol. Eugenol can impair the hardening of the composite and causes discoloration.

Storage

Store between 8°C / 46°F and 25°C / 77°F. If temperature exceeds this, storing in refrigerator is recommended.

Caution- Do not freeze, do not expose to direct light or other heat sources. Return to room temperature before use.

Keep material out of children's reach.

Lot Number and Expiry Date

The above are printed on each package and should be quoted in all correspondence which requires identification of the product.

Precautions

Use of protective gloves and a no-touch technique is recommended. If material contacts skin wash immediately with soap and water.

Emergency Measures

Ingestion-Seek immediate medical attention.

Contraindications

Application of the above materials is contraindicated: If a dry working field cannot be established. If a patient is known to be allergic to any of the ingredients.

For Professional Use Only

These Products may only be sold to dental professionals or other qualified persons, or following their instructions. These materials have been developed for use exclusively by dentists and must be handled according to the instructions of use. The recommendations included in the instructions correspond to the actual state of knowledge regarding dental techniques. İnci Dental cannot be held responsible for damages caused by other uses or incorrect handling.

Made in Türkiye

 **İncidental**
İnci Dental Tıbbi Malz. San. ve Tic. Ltd. Şti.
Deftardar Mah. Otakçılar Cd. E Blok Sit.
No:74/4 Eyüpsultan / İSTANBUL / TÜRKİYE
Tel: +90 212 632 63 45 Fax: +90 212 632 97 56
www.incidental.com.tr info@incidental.com.tr