

NIPRO

EN

NIPRO EXTENSION TUBE



Indications


This device is connected to an IV catheter, 3 way stopcock, AVF, PSV, or retention cannula

Single use only

Contraindications

The use of extension tube is not itself a therapeutic procedure and there are no particular contraindications when they are used for the intended purposes.

Shape and Structure

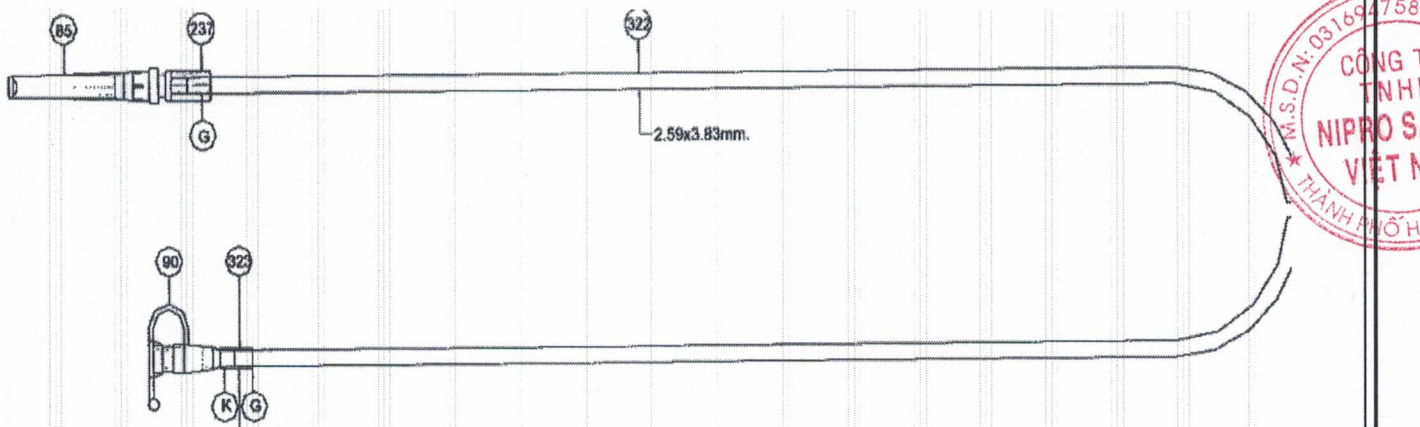

Description of components

Part No.	Name of Component
85	Needle protector
237	Lock joint T
322	Extension tube 2.59*3.83mm
323	Extension tube 3.38*4.22mm
90	PSV luer connector 1.8
G, K	-

Sterilized by ETO

Contents of unopened and undamaged package are Sterile, non-Toxic, non- Pyrogenic



Instructions for use

Extension sets are connected to IV catheters to allow a longer distance between the patient and the fluid being administered which is often necessary due to the size and volume of equipment being used at any one time.

Sets should be primed prior to connection with catheter, if it is being used for infusions, a syringe should be attached and the set primed before being connected to the patient.

Once an IV catheter has been introduced, the male luer lock of the extension tubing can be inserted into the port at the distal end of the catheter, sometimes this connection is secured by twisting a locking collar over the luer connection (provided on the extension set). The other end of the extension tubing can then be accessed by a syringe or more commonly, connected up to a primary giving set or syringe drive for the administration of fluids and medication.

Tube extensions with narrow bores are used for paediatrics and neonates because often a lower volume/flow rate is required for smaller patients.

Caution

Do not use if package is damaged

Discard after single use

Do not store at high temperature and humidity

Storage and shelflife

Storage: Avoid high temperature, humidity

Shelf-life

Refer to expiry date indicated on the packaging

Shelf-life: 5 years

NIPRO CORPORATION

3-9-3, Honjo – Nishi, Kita-Ku, Osaka, Japan



NIRPRO SALES THAILAND Co., LTD.,

2nd floor, Juthamard building, 89/170, Viphavadee Rangsit Road, Talad Bangkhen, Laski, Bangkok 10210, Thailand

NIPRO (THAILAND) CORPORATION LIMITED

10/2 Moo 8 Bangnomko, Sena Phra Nakorn Si Ayuthaya Thailand 13110

