

SINGLE USE SYSTEM
BREATHING SYSTEM INFORMATION

Jackson Rees System with Open Tail
2.1M Fresh Gas Line

QTY 1

REF 22JR0322M2L-OT

LOT XXXXXXXX

XXXX-XX

XXXX-XX



(01) 0 5055276 70372 3



(17) 000000 (11) 000000 (10) 0000000

USE ONLY FOR INTENDED PURPOSES



TLI_Iss_V03B PLI_Iss 01 Rev 00

CE
1639

INSTRUCTIONS: These directions are general guidelines intended for use by qualified medical staff.

1. If supplied, connect the expiratory (filter) end to the return of the appropriate equipment interface.
2. Connect inspiratory tubing to the outlet port on the appropriate interface.
3. If supplied, attach the breathing bag connection to the appropriate patient interface.
4. If supplied attach the patient elbow connector securely to the appropriate patient interface.
5. As required remove the selected gas sampling port and connect the appropriate gas sampling line.
6. If supplied connect machine end of sampling line to gas sampling monitor.

WARNINGS:

1. Follow the instructions provided by the equipment manufacturer for proper installation of the system.
2. Make sure all connections are secure.
3. **BEFORE USE:** test the system according to instructions provided by the manufacturer of the ventilator, gas sampling line and gas monitoring equipment.
4. Do not use in the presence of flammable anaesthetic agents.
5. Do not use if packaging is damaged.
6. For single patient use.
7. **HEATED WIRE SYSTEMS:** ensure correct heater is specified and check the connection is correct to the heater.
8. Entonox systems - if excessive resistance is felt on inhalation check for correct valve function.
9. **EXTRA LONG SYSTEMS:** any systems over 5m can cause flow and resistance changes. Please take this into account when using longer systems and setting your flow requirements.



Meditech Systems Ltd, Unit 3, Shrublands Estate, Sherstock, Shaftesbury, Dorset, SP7 9PT.
tel: +44(0)1747 821546 email: sales@meditechsystems.co.uk www.meditechsystems.co.uk

EC REP

Clowrey Consultancy, White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire,
Tel: +44(0)7888744362 E-mail: clowreyconsult@gmail.com