WOHLWEND AG

Vision

■ UNIVERSAL-STAINS

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## Instructions for use

- Vision Universal Stains & Shades consist of 8 stains and 4 basic colors A, B, C, D, (Shades)
  - They are available in pastes and powder form. The colors in powder form are ideal as a modifier for mixing in dentin and incisal masses (max. 5%)
  - By using the Vision-Universal Stains & Glaze, 3-di- mensional effects can be achieved even with mono- lithic restorations.
- ① Mix the stain liquid with a little glaze and apply thly to the ceramic surface.
  - ② Perform color matches and characterizations with Shades and Stains.
  - ③ Place ready-painted work on a firing tray and fire it with the appropriate program in the ceramic furna- ce without vacuum (see table „Firing instructions“)
  - ④ For fully anatomical zirconium and lithium disilicate restorations, an additionally glaze firing may be re- quired to achieve a consistently glossy surface .

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## Important

- To rewet dried Stains & Shades use the Vision Re-wetting Liquid!
- For all commercially available ceramic systems, the firing temperature for Vision Universal Stains & Glaze should be 10°C / 50°F lower than specified by the manufacturer.
- The firing parameters given in the table are standard values that must always be adapted to the particu- lar furnace used.