


Ningbo Greatcare Trading Co., Ltd.

Unit 93, Building 12, No. 818, Qiming Road,
Yinzhou, 315105 Ningbo, Zhejiang, China.

Tel: +86-574-8778 6632
Fax: +86-574-8778 6644
info@greatcare.cn

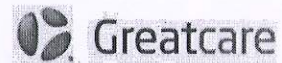

Creating a Better Care

IFU of Stomach Tube

INTENDED USE/ INDICATIONS

Stomach Tube is used to introduce food, nutrients, medication, or other material into the stomach for the comatose patients or the patients with dysphagia, or drain undesirable contents from the stomach, or decompress the stomach. The tube is inserted into a patient's stomach through the patient's nose or mouth.

CONTRAINDICATIONS

- Patients who with serious esophageal varices, nasal congestion, esophageal or cardia stricture/obstruction, serious dyspnea, erosive gastritis are not recommended to use the tube.
- The inflammation patients shall cautiously use.

WARNINGS/PRECAUTIONS

- Do not use if package has been opened or damaged.
- Read all warnings and instructions before use, improper use can result in serious or fatal illness or injury.
- Expert clinical judgment should be exercised in the selection of the appropriate size tube for each individual patient.
- Along one side of the nostrils and slowly inserted into the throat(14-16 cm), tell the patient to swallow and in the meantime put the tube down. If the patients experience symptoms such as cyanosis or cough during inserting, it should immediately stop the operation and pull out the tube.
- Once the tube has been passed, it should be checked to ensure proper placement in the Stomach.
- When the tube is lubricated well in advance of insertion, it is essential to verify that the lubricant has not entered and occluded the tube eyes thereby compromising tube function.
- Do not use after the expiry date.
- This is a disposable product for single use, any resterilization or reuse of the product can
- Precautions shall be taken when discarding device and disposal of the device shall be made in accordance with applicable national regulations for biological hazardous waste.

ADVERSE REACTIONS

Reported adverse reactions include the following : acid reflux with subsequent esophageal stricture, aspiration pneumonia, dehydration, distension, dry mouth and lips, electrolyte imbalance, esophagitis, sore throat, tracheal intubation and trauma to mucosa of the pharynx-esophagus-stomach ect.

DIRECTIONS FOR USE

- Peel off the package, and take out the tube.
- Insert the tube through the oral cavity to stomach. Meanwhile pay attention to the mark on the tube to insure the insertion position. If necessary, use radiograph to insure the insertion position.
- Connect the connector with the suction device.
- Adjust the pressure, move the respiration machine to suck contains in stomach.

CAUTION:

- Single use only.



- Sterile if package is unopened and undamaged.
- Store in a dark dry place.



Keep dry



Use by date



Manufacturing date



Batch Code



Do not resterilize



Manufacturer



Do not expose to sunlight



Re-use is not allowed



Not to be used in case package is damaged

