



Document Information:

Document Author: Maschat, Cordula {DOMC~Mannheim}
Business Area / Unit: Roche Professional Diagnostics
Confidentiality: Confidential
Document Class: Device Master Record
Document Type: Device Master Record
Document Creator: Maschat, Cordula {DOMC~Mannheim}
Document Lifecycle Status: Signed
Valid From: 12-Oct-2018 04:17:55 (UTC)
Valid To:
Document Title: C_0201_RSP_03333701000
Document Number: 0000000000001008018000550
Document Version: 03
Template: No

Global Group: Regulatory Affairs
Global SubGroup: Specification Prod. (Registr.)
Local Group: Reg. Specification
Language: English
Site: RDG Germany
Department: C_DOMCC
Document Applies To: C_QC Centralized Assays
Document Description: Alkaline phosphatase acc. to IFCC Gen.2 (ALP2) - "Only for use
in Regulatory Affairs"
Legacy Document Number: C_0201_RSP_03333701000_03333752000
Change Request eCCP PLM: 400037429

Electronic Signatures:

Signed By: maschac1 (Cordula Maschat {DOMCSD..6164})
Role: Reviewer
Signature Differentiation: Quality Control
Signed Date: 11-Oct-2018 06:41:41 (UTC)

Signed By: loewh (Heiner Loew {DOMC....6164})
Role: Approver
Signature Differentiation: Quality Control
Signed Date: 11-Oct-2018 07:16:58 (UTC)



Specification

Product: **Alkaline phosphatase acc. to IFCC Gen.2
(ALP2)**

Catalog No. 03333701190
03333752190


Intended use:

In vitro test for the quantitative determination of alkaline phosphatase in human serum and plasma on Roche/Hitachi cobas c and COBAS INTEGRA systems.

Test principle:

Colorimetric assay in accordance with a standardized method.

In the presence of magnesium and zinc ions, p-nitrophenyl phosphate is cleaved by phosphatases into phosphate and p-nitrophenol.

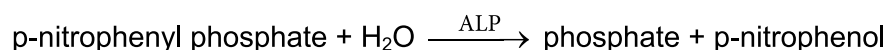

The p-nitrophenol released is directly proportional to the catalytic ALP activity.
It is determined by measuring the increase in absorbance.

**Roche Diagnostics GmbH
Roche Diagnostics Global Operations
Quality Control Mannheim**

D-68305 Mannheim
Phone: +49-621-759-0
Fax: +49-621-759-2890

1. Reagents-working solutions:

1.1 Activator solution (B):

1.1.1 pH-value (30 °C): 10.39 - 10.49

1.2 p-Nitrophenyl phosphate solution (C):

1.2.1 pH-value (25 °C): 8.40 - 8.60

2. Performance test:

2.1 Recovery on COBAS INTEGRA analyzer(s):

PreciControl ClinChem Multi 1 (Cat. No. 05947626190): 90 - 110 %

PreciControl ClinChem Multi 2 (Cat. No. 05947774190): 90 - 110 %

2.2 Recovery on Roche/Hitachi cobas c analyzer(s):

PreciControl ClinChem Multi 1 (Cat. No. 05947626190): 90 - 110 %

PreciControl ClinChem Multi 2 (Cat. No. 05947774190): 90 - 110 %

Also required:

Calibrator f.a.s. (Cat. No. 10759350190)