



**JOIN MD VIET NAM CO., LTD**

## **INSTRUCTION**

### **OPTIMA SPLINT (ROLL)**

#### **1. Point**

The product is used for temporary fixation of fractures, sprains and deformities; Orthopedic molding in other orthopedic cases.

#### **2. Ingredients**

Fiberglass molded fabric, polyurethane impregnated polyester fabric

#### **3. Preservation**

Avoid exposing the product to the outside air.

#### **4. Instruction**

- Select the OptimaSplint product according to the desired size and take it out of the silver pouch.
- Dip the product into cool water (18oC - 30oC), squeeze 2-3 times under water for the product to fully absorb.
- Roll or fold the product and wring it out.
- Place product on towel then roll to fit and wring to remove residual water.
- Stick the product on the desired area of the patient and wrap it with an elastic band. At this time, the molding and shaping will be done. Setting time is approximately 3-4 minutes. After 20 minutes, the bandage will be completely fixed.

#### **5. Warnings and cautions**

- There have been no reports of adverse reactions associated with the use of the product.
- Consult your doctor before use.

**JOIN MD VIET NAM CO., LTD**

**DIRECTOR**



**JANG KYOUNGHO**