


SURGICAL SHEETS

Instructions For Use

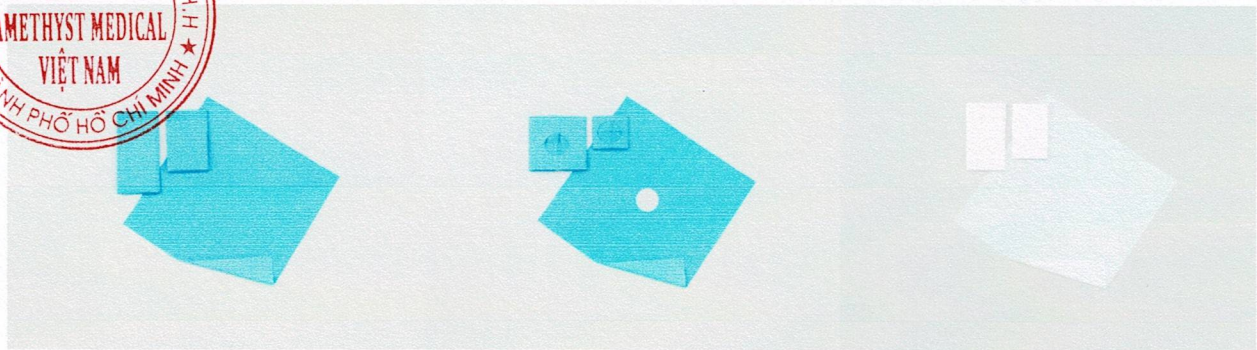

#814

#818

#914

PRODUCT NAME: Surgical Sheets

I. INTENDED USE

Spread on the hospital bed, examination table, instrument table or spread over other items to absorb liquids (fluid, blood, chemicals) to avoid infection in medical facilities.

II. CONTRAINDICATIONS

None.

III. PRECAUTIONS AND WARNINGS

- When the sheet is too wet, it is necessary to change another sheet to avoid skin inflammation for the patient.
- Areas of the body that are prone to sores should avoid contact with damp bed sheets.
- Do not let it get wet or damp.
- Store in a cool place, avoid direct sunlight and high temperature, avoid contact with chemicals.

IV. INSTRUCTIONS FOR USE

- Spread the sheet flat on the operating table, hospital bed, instrument table... adjust the position to suit the intended use.
- Single-use products, after use need to be disposed of according to regulations on medical waste treatment.



Cobes Industries (Hefei) Co., Ltd.

No. 39 Liaoyuan Road. Xingcheng Development Zone, Hefei, China 231600