

## Notification


Date: March 25<sup>th</sup>, 2022







To whom it may concern,

We, A.R. Medicom Inc. (Taiwan) Ltd. located in 4F.-3, NO. 163, Sec.1, Keelung Road, Xinyi Dist, Taipei City, Taiwan, hereby certify that the Instruction for Use below is active for Medicom's product as below:

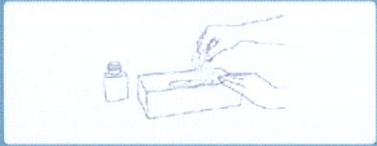


- 1174 Nitrile Glove (White)
- 1175 Nitrile Glove (Blue)
- 1188 Latex Glove (Powder-Free)
- 1205 Latex Glove (Powered)

**How to wear Gloves**



-   
**1** Take out the first glove
-   
**2** Pick up the glove by holding the edge of the wrist part.
-   
**3** Put on the glove
-   
**4** Take out the second piece of glove
-   
**5** Move your second hand to fit into the glove; this way, it can avoid contamination from touching the forearm.
-   
**6** Do not touch any unintended items aside from the target items

**How to Take off Gloves**

-   
**1** Pick the front wrist part to remove the glove, inside out.
-   
**2** Hold the removed piece with the hand in glove. Slip your bare hand into the space between your wrist and glove to remove it inside out.
-   
**3** Discard the gloves

Best Regards,



A.R. Medicom Inc. (Taiwan) Ltd.



## CÔNG TY TNHH TRANG THIẾT BỊ NHA KHOA

ĐC: 199 Nguyễn Thị Minh Khai, P. Nguyễn Cư Trinh, Quận 1, Tp. Hồ Chí Minh  
MST: 0302130894

# THÔNG BÁO

Ngày: 25/03/2022

Gửi các cơ quan chức năng:

Chúng tôi, **A.R. Medicom Inc (Taiwan) Ltd**, trụ sở tại 4F.-3, No.163, SEC.1, Keelung Road, Xinyi Dist, Taipei city, Taiwan, xác nhận rằng Hướng dẫn sử dụng bên dưới được sử dụng cho những sản phẩm Medicom bên dưới:

- 1174 Nitrile Glove (White)
- 1175 Nitrile Glove (Blue)
- 1188 Latex Glove (Powder –Free)
- 1205 Latex Glove (Powdered)

### Hướng dẫn sử dụng:

#### a. Cách đeo găng tay:

1. Lấy găng tay đầu tiên.
2. Cầm đầu mép găng tay.
3. Đeo găng vào tay.
4. Lấy găng tay thứ hai, thực hiện giống bước 2,3.
5. Di chuyển tay để vừa với găng tay. Bằng cách này, nó có thể tránh nhiễm bẩn từ cánh tay chạm vào.
6. Không chạm vào bất kỳ vật nào khác ngoài những vật dụng trong mục đích sử dụng.

#### b. Cách tháo găng tay:

1. Cầm phần mép găng tay từ mặt trước cổ tay để tháo găng, từ trong ra ngoài.
2. Giữ găng tay vừa tháo bằng tay đang đeo găng. Luồn tay trần vào khoảng giữa cổ tay và găng tay của bàn tay đang đeo găng để tháo găng từ trong ra ngoài.
3. Bỏ găng tay vừa tháo vào thùng rác.

**Người đại diện hợp pháp của cơ sở**  
Ký tên (Ghi họ tên đầy đủ, chức danh)  
Xác nhận bằng dấu hoặc chữ ký



**GIÁM ĐỐC**

*Chiều Thị Mỹ Châu*