## Allergy Relief

100% NATURAL RELIEF FROM ALLERGIC RHINITIS SYMPTOMS

HYPERTONIC SEA WATER SOLUTION

ALGOMER™ COMPLEX + DEXPANTHENOL

**EN** Sinomarin® Plus Algae Allergy Relief with Algomer™ complex is a 100% natural spray that offers relief from nasal symptoms associated with seasonal or chronic allergies. Its unique –patent pending– formulation combines the natural decongestant action (osmosis effect\*) of hypertonic sea water (2.3% NaCl) with the well-documented properties –beneficial to the nasal mucosa– of seaweed (algae) extracts\*\* present in Algomer™ complex, in addition to Dexpantenol. Sinomarin® Allergy Relief cleanses the nasal cavities helping to eliminate allergens (mechanical action) that may worsen allergic rhinitis symptoms.

The seawater used in Sinomarin® Allergy Relief is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents, and provides sea water of outstanding quality, rich in the sea's mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

**Algomer™ complex** is a unique combination of 100% natural ingredients extracted from *Undaria pinnatifida* and *Spirulina platensis* algae (seaweed). Both algae have long been used in Oriental medicine to promote health and treat multiple conditions, and are part of the Far-Eastern diet. Formulated specifically for nasal use, **Algomer™ complex** is composed of a blend of natural algae components, namely, phycocyanin and a series of polysaccharide complexes including fucoidans. These components form a protective layer that encapsulates, traps and prevents contact and/or entry of harmful external agents (e.g. allergens, pollutants, viruses, bacteria, etc.) within the nasal mucosa, helping to reduce symptoms of allergic rhinitis and strengthen its natural function. *Undaria pinnatifida* and *Spirulina platensis* algae are harvested from the most pristine waters of Patagonia in Argentina and Brittany in France, respectively, and are certified organic.

**Dexpantenol** –also known as provitamin B5– is a 100% natural ingredient well-known for its soothing and moisturizing properties. It relieves itching and local irritation and moisturises the nasal mucosa.

Sinomarin® Allergy Relief can be used alone, as a natural alternative to anti-allergic treatments (local corticosteroids, antihistamines) or in combination with them to help enhance their efficacy.

#### Properties:

- Relieves allergic rhinitis symptoms including nasal congestion, sinus pressure, runny nose and sneezing.
- Relieves local irritation and soothes the nasal mucosa.
- Cleanses the nasal cavities eliminating allergens.
- Moisturises the nasal mucosa, relieving nasal dryness.
- Helps promote natural recovery of the nasal mucosa.

#### Sinomarin® Allergy Relief is recommended:

- For the natural relief of nasal symptoms associated with seasonal (e.g. hay fever) or chronic allergies such as blocked nose, sinus pressure, runny nose, sneezing, itchy nose, local irritation.
- To eliminate mucus and mucus-trapped allergens (mechanical action) that may worsen allergic rhinitis symptoms.
- In cases where prescribed anti-allergic agents are contraindicated.
- To help enhance the efficacy of prescribed anti-allergic drugs.

**Composition:** Sea water (equivalent to 2.3% NaCl), Purified water, Algomer™ complex (*Undaria pinnatifida* & *Spirulina platensis* extracts), Dexpantenol.

Sinomarin® Allergy Relief is free of antihistamines & steroids, preservatives and chemical additives. 100% natural. Does not cause drowsiness.

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**Allergy Relief**  
100% NATURAL RELIEF FROM  
ALLERGIC RHINITIS SYMPTOMS

**HYPERTONIC SEA WATER SOLUTION**  
**ALGOMER™** + DEXPANTHENOL  
**COMPLEX**

**Recommended dosage:**

For adults & children over 6 years. Use as per below, or as per physician's instructions. May be used without interruption for up to 30 days.

6-12 years: 1 to 2 sprays in each nostril up to 3 times daily.

>12 years: 2 to 4 sprays in each nostril up to 5 times daily.

**Precautions:** For nasal use only. Do not use same nasal applicator on more than one individual. Do not use past expiry date. Keep out of reach of children. Do not use in cases of sensitivity or if allergic to any of the components.

**Advice:** A mild and temporary irritation may be experienced at the beginning of use.

**Storage conditions:**

Store below 25 °C. Protect from sunlight.

**Other information:**

Special valves prevent contamination of the solution once the spray has been opened and thereby the necessity to use preservatives.

Sinomarin® is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

Sinomarin® products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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- Blow nose to clear out mucus before using Sinomarin® Allergy Relief.
- Remove protective cap.
- Hold the spray bottle upright and spray a couple of times in order to activate the spray pump.
- Bend neck gently forward over a wash basin and tilt head to the right.
- Gently insert applicator into the left nostril, in parallel with the nasal septum, and press firmly.
- Turn head to opposite side and repeat application in other nostril.
- Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- Repeat procedure if necessary.
- Wash applicator well with warm water and wipe dry after each use.

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\* Natural process characterized by water movement out of the swollen nasal tissues due to the local difference in salt (NaCl) content induced by nasal application of Sinomarin®.

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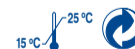
www.sinomarin.com

ITEM REFERENCE CODE	213621D
ITEM DESCRIPTION	INSERT SINOMARIN PLUS ALGAE ALLERGY RELIEF 30ml ENG-FR-GR
LATEST UPDATE OF CODE/ DATE	8/7/2020
VERSION DATE & TIME	10/09/2020 - 13:10
APPROVAL BY GI REGULATORY/ DATE, NAME & SIGNATURE	
APPROVAL BY GI MARKETING/ DATE, NAME & SIGNATURE	
COMMENTS	

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Sinomarin® is a registered trademark for Gerolymatos International S.A., E.U.
Sinomarin® est une marque déposée par Gerolymatos International S.A., U.E.
Gerolymatos International S.A.