Cold & Flu Relief

100% NATURAL RELIEF FROM NASAL SYMPTOMS OF THE COMMON COLD & FLU

HYPERTONIC SEA WATER SOLUTION

ALGOMER™
COMPLEX

+ EUCALYPTUS,
SPEARMINT & THYME

EN Sinomarin® Plus Algae Cold & Flu Relief with Algomer™ complex is a 100% natural spray that offers relief from nasal symptoms associated with the common cold and flu. Its unique –patent pending– formulation combines the natural decongestant action (osmosis effect*) of hypertonic sea water (2.3% NaCl) with the well-documented properties –beneficial to the nasal mucosa– of seaweed (algae) extracts** present in **Algomer™ complex**, in addition to the essential oils and extracts of Eucalyptus, Spearmint and Thyme. **Sinomarin® Cold & Flu Relief** cleanses the nasal cavities helping to eliminate infectious agents (mechanical action).

The seawater used in **Sinomarin® Cold & Flu Relief** is collected from Cancale Bay at the coast of Brittany in France. The area is well-known for its open seas and strong tidal currents, and provides sea water of outstanding quality, rich in the sea's mineral salts and trace elements. The sea water quality is continuously monitored by the French Research Institute for Exploitation of the Sea (IFREMER) under the auspices of the French Government.

Algomer™ complex is a unique combination of 100% natural ingredients extracted from *Undaria pinnatifida* and *Spirulina platensis* algae (seaweed). Both algae have long been used in Oriental medicine to promote health and treat multiple conditions, and are part of the Far-Eastern diet. Formulated specifically for nasal use, **Algomer™ complex** is composed of a blend of natural algae components, namely, phycocyanin and a series of polysaccharide complexes including fucoidans. These components form a protective layer that encapsulates, traps and prevents contact and/or entry of harmful external agents (e.g. viruses, bacteria, etc.) within the nasal mucosa, helping to relieve symptoms of the common cold and flu and strengthen its natural function. *Undaria pinnatifida* and *Spirulina platensis* algae are harvested from the most pristine waters of Patagonia in Argentina and Brittany in France, respectively, and are certified organic.

The essential oils of **Eucalyptus globulus** & **Mentha spicata** and **Thymus vulgaris** extract are rich in ingredients (e.g. 1,4-cineole, menthol, carvacrol, etc.) well-known for their benefits on the nasal mucosa and their ability to provide a refreshing sensation. **Sinomarin® Cold & Flu Relief** also contains **Glycerin** offering additional moisturizing properties to the irritated nasal mucosa. All components are 100% natural.

Sinomarin® Cold & Flu Relief can be used alone, as a natural alternative to medicated decongestants (local corticosteroids, antihistamines, vasoconstrictors), or in combination with them to help enhance their efficacy.

Properties:

- Relieves symptoms of the common cold & flu including nasal congestion, sinus pressure and runny nose.
- Fluidifies mucus secretions.
- Cleanses the nasal cavities removing mucus and mucus-trapped infectious agents (via mechanical action).
- Moisturizes the nasal mucosa and relieves nasal dryness.
- Helps promote natural recovery of the nasal mucosa.

Sinomarin® Cold & Flu Relief is recommended:

- For the natural relief of nasal symptoms associated with the common cold & flu (e.g. nasal congestion, runny nose) and in cases of rhinosinusitis.
- To eliminate infectious agents that may lead to secondary infections e.g. sinusitis, otitis (via mechanical action).
- In cases where medicated decongestants are contraindicated (e.g. high blood pressure, heart conditions, diabetes).
- To help enhance the efficacy of medicated decongestants.

Composition: Sea water (equivalent to 2.3% NaCl), Purified water, Algomer™ complex (*Undaria pinnatifida* & *Spirulina platensis* extracts), *Thymus vulgaris* extract, *Eucalyptus globulus* essential oil, *Mentha spicata* essential oil, Glycerin of vegetal origin, Propanediol.

Sinomarin® Cold & Flu Relief is free of drugs, preservatives and chemical additives. 100% natural.

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Cold & Flu Relief

100% NATURAL RELIEF FROM
NASAL SYMPTOMS OF THE COMMON COLD & FLU

HYPERTONIC SEA WATER SOLUTION

ALGOMER™ + EUCALYPTUS,
COMPLEX SPEARMINT & THYME

Recommended dosage:

For children over 6 years and adults. Use as per below, or as per physician's instructions. May be used without interruption for up to 30 days.

6-12 years: 2 to 3 sprays in each nostril up to 3 times daily.

>12 years: 2 to 4 sprays in each nostril up to 5 times daily.

Precautions: For nasal use only. Do not use on more than one individuals. Do not use past expiry date. Keep out of reach of children. Do not use during pregnancy or breastfeeding. Do not use in cases of sensitivity or if allergic to any of the components.

Advice: A mild and temporary irritation may be experienced at the beginning of use.

Storage conditions:

Store below 25°C. Protect from sunlight.

Other information:

Special valves prevent contamination of the solution once the spray has been opened and thereby the necessity to use preservatives.

Sinomarin® is a complete line of hypertonic sea water solutions covering the needs of all ages from newborns to adults.

Sinomarin® products are CE-marked Medical Devices conforming to the requirements of Directive 93/42 of the European Union.

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- 1 Blow nose to clear out mucus before using **Sinomarin**® Cold & Flu Relief.
- 2 Remove protective cap.
- 3 Hold the spray bottle upright and spray a couple of times in order to activate the spray pump.
- 4 Bend neck gently forward over a wash basin and tilt head to the right.
- 5 Gently insert applicator into the left nostril, in parallel with the nasal septum, and press firmly.
- 6 Turn head to opposite side and repeat application in other nostril.
- 7 Remove spray from nostril, return to upright position while allowing solution to work for a few seconds and then blow the nose.
- 8 Repeat procedure if necessary.
- 9 Wash applicator well with warm water and wipe dry after each use.

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* Natural process characterized by water movement out of the swollen nasal tissues due to the local difference in salt (NaCl) content induced by nasal application of Sinomarin®.

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** Φαιοφύκη: *Undaria pinnatifida*, κυανοφύκη: *Spirulina platensis*.

Sinomarin® Cold & Flu Relief is covered by an international patent application (pending) filed by Gerolymatos International S.A., E.U.

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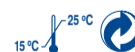
www.sinomarin.com

ITEM REFERENCE CODE	213622D
ITEM DESCRIPTION	INSERT SINOMARIN PLUS ALGAE C&F RELIEF 30ML ENG-FR-GR
LATEST UPDATE OF CODE/ DATE	8/7/2020
VERSION DATE & TIME	28/08/2020-13:34
APPROVAL BY GI REGULATORY/ DATE, NAME & SIGNATURE	
APPROVAL BY GI MARKETING/ DATE, NAME & SIGNATURE	
COMMENTS	

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Sinomarin® is a registered trademark for Gerolymatos International S.A., E.U.
Sinomarin® est une marque déposée par Gerolymatos International S.A., U.E.