
## EVACUATED BLOOD COLLECTION TUBES

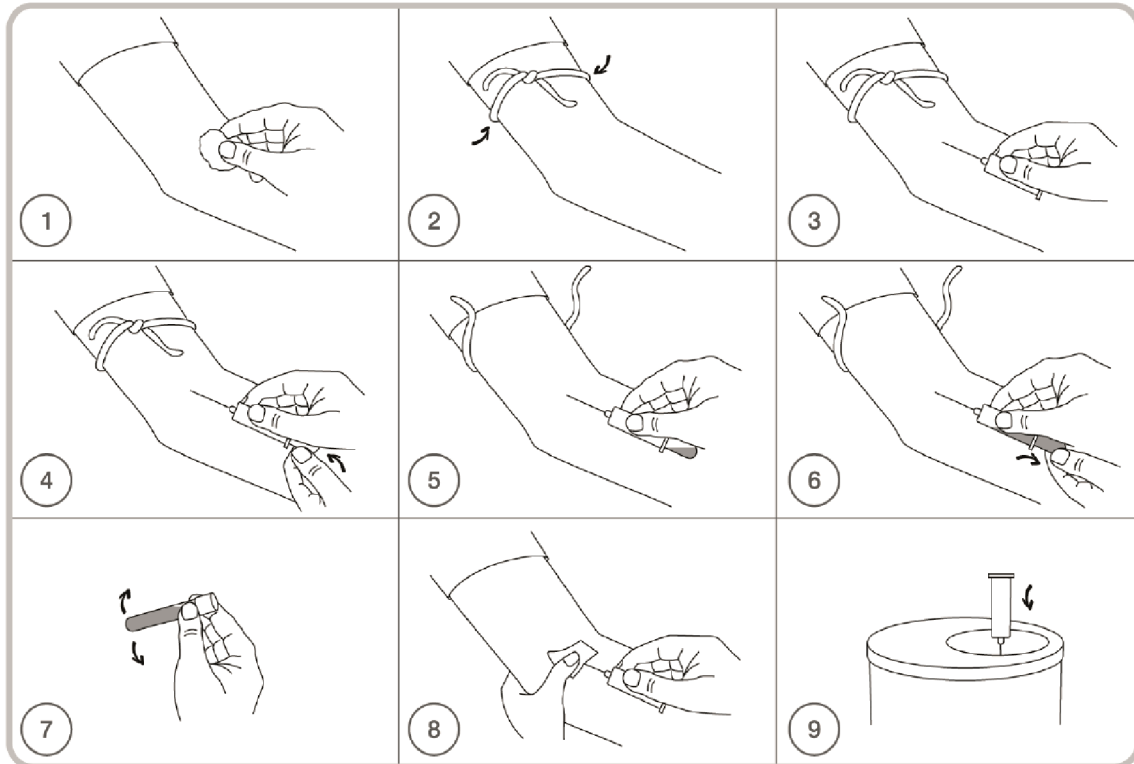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### GENERAL WARNING USE

Wear gloves during venipuncture and when handling blood collection tubes. Do not use the product if it is expired or damaged. Do not use the product for any other use which would be considered improper. Store in a dry place, away from sources of heat and direct sunlight. The device is disposable, the reuse of the device will lose all performances and features. Dispose the product in accordance with current legislation in force.

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General instructions (see picture):

1. Select the tube appropriate for the required specimen and prepare the appropriate blood collection needle. Place the patient's arm in a downward position and prepare the venipuncture site with an appropriate antiseptic;
2. Apply the tourniquet above the puncture site;
3. Perform the venipuncture;
- 4-5. Push the tube into the holder and onto the needle valve puncturing the rubber diaphragm. Center the tubes in the holder when penetrating the cap to prevent sidewall penetration and subsequent premature vacuum loss. Remove the tourniquet as soon as blood appears in the tube;
6. When the first tube is filled and the blood flow ceases, gently remove it from the holder;
7. Gently invert each tube immediately as it is removed from the holder, using the correct number of inversions to achieve the proper mix of additive and blood.
8. As soon as the blood stops flowing in the last tube, remove the needle from the vein. Once clotting has occurred, apply a bandage, if desired;
9. Dispose of the used needle device in an approved biohazard container.

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## Letter of Confirmation

To: Whom it may concern

We, **F.L. Medical s.r.l.** based at **Via Enrico Mattei 20 - 35038 Torreglia (PD) Italy**, hereby confirm that the document AL402G Rev 04 date 03/05/2021 is applicable to all Vacumed® test tubes for blood collection, included the products below:

STT	Code	Product name
1	42010	Vacumed® 13x75 mm with K2 EDTA x 2 ml of blood, lavender cap, sterile
2	42011	Vacumed® 13x75 mm with K2 EDTA x 3 ml of blood, lavender cap, sterile
3	42012	Vacumed® 13x75 mm with K2 EDTA x 4 ml of blood, lavender cap, sterile
4	43016	Vacumed® 13x100 mm with K2 EDTA x 6 ml of blood, lavender cap, sterile
5	44019	Vacumed® 16x100 mm with K2 EDTA x 9 ml of blood, lavender cap, sterile
6	42110	Vacumed® 13x75 mm with K3 EDTA x 2 ml of blood, violet cap, sterile
7	42111	Vacumed® 13x75 mm with K3 EDTA x 3 ml of blood, violet cap, sterile
8	42112	Vacumed® 13x75 mm with K3 EDTA x 4 ml of blood, violet cap, sterile
9	43116	Vacumed® 13x100 mm with K3 EDTA x 6 ml of blood, violet cap, sterile
10	44119	Vacumed® 16x100 mm with K3 EDTA x 9 ml of blood, violet cap, sterile
11	42310	Vacumed® 13x75 mm with Lithium Heparin x 2 ml of blood, dark green cap, sterile
12	42311	Vacumed® 13x75 mm with Lithium Heparin x 3 ml of blood, dark green cap, sterile
13	42312	Vacumed® 13x75 mm with Lithium Heparin x 4 ml of blood, dark green cap, sterile
14	43316	Vacumed® 13x100 mm with Lithium Heparin x 6 ml of blood, dark green cap, sterile
15	44319	Vacumed® 16x100 mm with Lithium Heparin x 9 ml of blood, dark green cap, sterile
16	42210	Vacumed® 13x75 mm with Potassium Fluoride + K3 EDTA x 2 ml of blood, grey cap, sterile
17	42211	Vacumed® 13x75 mm with Potassium Fluoride + K3 EDTA x 3 ml of blood, grey cap, sterile

18	42212	Vacumed® 13x75 mm with Potassium Fluoride + K3 EDTA x 4 ml of blood, grey cap, sterile
19	43216	Vacumed® 13x100 mm with Potassium Fluoride + K3 EDTA x 6 ml of blood, grey cap, sterile
20	44219	Vacumed® 16x100 mm with Potassium Fluoride + K3 EDTA x 9 ml of blood, grey cap, sterile
21	42410	Vacumed® 13x75 mm with Sodium Citrate 3,2 % x 1,8 ml of blood for coagulation, light blue cap, sterile
22	42411	Vacumed® 13x75 mm with Sodium Citrate 3,2 % x 2,7 ml of blood for coagulation, light blue cap, sterile
23	42610	Vacumed® 13x75 mm with Clot Activator x 2 ml of blood, red cap, sterile
24	42611	Vacumed® 13x75 mm with Clot Activator x 3 ml of blood, red cap, sterile
25	42612	Vacumed® 13x75 mm with Clot Activator x 4 ml of blood, red cap, sterile
26	43616	Vacumed® 13x100 mm with Clot Activator x 6 ml of blood, red cap, sterile
27	44619	Vacumed® 16x100 mm with Clot Activator x 9 ml of blood, red cap, sterile
28	42721	Vacumed® 13x75 mm with Gel Separator + Clot Activator x 3,5 ml of blood, golden-yellow cap, sterile
29	43726	Vacumed® 13x100 mm with Gel Separator + Clot Activator x 5 ml of blood, golden-yellow cap, sterile
30	44728	Vacumed® 16x100 mm with Gel Separator + Clot Activator x 8 ml of blood, golden-yellow cap, sterile

Signature:

Name: Alessandro Fiore

Position: Quality Assurance Manager

For and on behalf of **F.L. Medical s.r.l.**