



www.ssnmedical.com

SSN MEDICAL PRODUCTS SDN BHD

(Company Registration No.441893-K) (GST ID No : 000401211392)

Lot 31793 (formerly Lot PT 7177), Jalan KPB 12, Kampung Baru Balakong,
43300 Seri Kembangan, Selangor Darul Ehsan, Malaysia.

Tel : +603. 8961 6869 (Main Line) Fax : +603. 8961 8829

Email : information@ssnmedical.com

DIRECTIONS FOR USE

Latex Examination Gloves Prepowdered / Powder-Free

1. Preparations for use : Ensure the right selection gloves with correct size.
2. How to use
 - Dry hands before donning the gloves.
 - Take out the product after checking the package.
 - Don the gloves carefully. Do not apply excessive force to pull the gloves when donning.
 - Discard the gloves with an appropriate waste disposal container after use according to local disposal regulations.
 - Keep the fingernails short.
 - Change gloves frequently to avoid tearing issue.
3. Cautions for use
 - The product is non-sterile and for single use only.
 - After donning, remove powder by thoroughly wiping gloves with a sterile wet towel or other effective method.
 - This product contains natural rubber latex which may cause allergic reactions.
 - Store away from sources of heat and direct sunlight.