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**VENOUS SIDE: BLUE**  
*TĨNH MẠCH: XANH*

### CAUTIONS:

- SINGLE USE ONLY: Discard after use.
- Do not use it if any breakage in the packing of the sterilization bag or any defects on the sterilization bag are found.
- Always check that there are no errors or failure during extra-corporeal circulation. If an air bubble or error is found in the blood circulation system, follow doctor's instruction and take appropriate action.
- When collecting blood from or infusing drug into the rubber button, do not use a needle with a thickness of 21 gauge or larger.
- Keep away from getting wet. Do not store the sterilization bag in direct sunlight, at high temperature or humidity.
- This blood tubing line is sterilized by steam sterilization, so small dark brown parts or dented parts of the tubing line may be observed, these do not interfere with its function.
- Setting the blood tube on a blood pump, adjust the rollers of blood pump to press the tube just as strong as keeping 1.5m water height.

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Manufactured by: **NIKKISO VIETNAM MFG CO., LTD.**  
Road 19, Tan Thuan EPZ, District 7, HO CHI MINH CITY, VIETNAM. Tel.: 872 3320 (4 lines).  
Investment license no: 081-GPDT dated on 11-3-1996

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