

Sterile Hypodermic Needles for Single Use instructions

- 1, Please check the packaging integrity before use, damaged packaging, sheath fall off, and abnormal needlepoint foreign body in it, more than the validity, it is prohibited to use;
- 2, Open the small package, should prevent products fall to the ground;
- 3, Operation is prohibited by hand or not sterilized items contact needle, needle used shall not exceed 20° bend.
- 4, This product can only be a one-time use, don't repeat use, destroyed after use; 5, Away from chlorine and other corrosive disinfection storage;
- 6, Operating norms related to the products use must conform to the medical department and related regulatory requirements, limited to the training of doctors or nurses.