

## Breathing System Filters, Single Use, Disposable

### Explanation of symbols;

	= Single Use. The definition of single use is for use on a single patient for a single procedure.		= Manufacturer
	= Do not reprocess / re-sterilise		= Temperature range
	= Use by date		= Consult instructions for use
	= Lot or batch number.		= Keep away from sunlight
	= Date of manufacture.		= Keep dry
	= Device catalogue number / product code / stock		= Contains latex
	= Caution		= Do not use of packaging is wet or damaged.
			= Contains or Presence of Phthalates.

### Instructions for use

1. This product may only be used by, or under the supervision of, healthcare professionals
2. All connections should be securely attached, by using a twisting action, to ensure a leak free fit.
3. All patient circuits must be leak tested prior to patient connection.
4. The in-use life of the filter is: -
  - a. Inspiratory use in normal conditions Electrostatic = 72 hours , HEPA = Max 48 hours
  - b. Expiratory and inspiratory in damp or humid conditions Electrostatic = 48 hours , HEPA = MAX 12 HOURS
  - c. Use with active humidification Electrostatic = 24 hours
  - d. **HEPA filters should not be used with active humidification !**
5. The Luer lock connection is for gas sampling only
6. When the Luer lock connection is not being used ensure that the cap is securely fitted to prevent leakage from the filter.

### Warnings

- This product should not be used if the integrity of the packaging has been compromised.
- Do not use if the packaging is contaminated or wet.
- Do not use if the product has exceeded the shelf life as printed on the product label
- This product is for **single use only**.
- The product must not be reprocessed.
- The product must not be altered or dismantled.

### Disposal

These devices should be disposed of following the local rules for disposal of contaminated clinical waste.

**Ordering information:**

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

E mail; [orders@meditechsystems.co.uk](mailto:orders@meditechsystems.co.uk)

Tel: +44(0)1747 821546 Fax: +44(0)1747 825038

**EU Authorised Representative:**

Clowrey Consultants, White House at Bridge, Holycross, Thurles, C Tipperary, Eire

E mail; [clowreyconsult@gmail.com](mailto:clowreyconsult@gmail.com) Tel; +44(0)7888744362