Medtronic

Medtronic Burs

Fraises Medtronic / Frese Medtronic /
Knochenfräsen von Medtronic / Fresas
Medtronic / Medtronic boren / Medtronic
bor / Medtronic -poranterät / Medtronic
borr / Brocas Medtronic / Γλύφανα
Medtronic / Wiertła Medtronic / Frézy
Medtronic / Medtronic fúrók / Medtronic
frezler / Borene fra Medtronic

Product Information & Instructions

Informations et instructions concernant le produit /Informazioni
sul prodotto ed istruzioni per l'uso /Produktinformation und
Gebrauchsanweisung /Instrucciones e información sobre el producto
/ Productinformatie en instructies / Produktoplysninger og vejledning
/ tuotetta koskevat tiedot ja käyttöohjeet / Produktinformation och
instruktioner / Informação do produto e instruções / Πληροφορίες
προϊόντος και οδηγίες / Informacje o produkcie i instrukcje / Údaje o
výrobku a pokyny / Termék információ és használati utasítás / Ürün
Bilgileri ve Talimatlar / Produktinformasjon og instruksjoner

Rx Only

SYMBOLS

LONG

Long /Long/Lungo/Lang/Largo/Lang/Lang/Pitkä/Lång/Comprido/
Επίμηκες/Długie/Dlouhé/Hosszú/Lang/Uzun/

MEDIUM

Medium /Moyen/Medio/Mittel/Mediano/Medium/Medium/
Keskikokoinen/Medellång/Médio/Mέσο/Średnie/Średni/Közepes/
Medium/Orta/

STANDARD

Standard /Standard/Standard/Standard/Estándar/Standaard/Standard/
Standardi/Standard/Padrão/Τυπικό/Standardowe/Standardni/Szabványos/
Standard/Standart/

FINE

Fine /Fin/Fine/Fein/Fino/Fijn/Fin/Siro/Fin/Fino/Λεπτόκοκο/Drobne/
Jemné/Finom/Fin/İnce/

COARSE

Coarse /Grossier/Ruvido/Grob/Grueso/Grof/Grov/Karkea/Grov/Grosso/
Αδρόκοκο/Grube/Hrubé/Durva/Grov/Kalın/

CUTTING

Cutting/Coupant/Da taglio/Schneidend/Cortadora/Snijdend/Skærende/
Leikkaava/Skärande/Cortante/Κοπή/Cięcie/Ostré/Forgácsoló/Skjærende/
Kesme

STEEL

Steel /Acier/Acciaio/Stahl/Acero/Staal/Stål/Τεράς/Stål/Aço/Ατσάλι/Stal/
Ocel/Acél/Stål/Çelik/

STAINLESS STEEL

Stainless Steel /Acier inoxydable/Acciaio inossidabile/Rostfreier Stahl/
Acero inoxidable/Roestvrij staal/Rustfrit stål/Ruostumaton teräs/Rostfritt
stål/Aço inoxidável/Ανοξείδωτο ατσάλι/Stal nierdzewna/Nerezová
ocel/Rozsdamentes acél/Rustfritt stål/Paslanmaz Çelik/

For use up to “NNNN” RPM. “NNNN” rpm au maximum | Da usare
fino a “NNNN” giri al minuto | Bis zu einer Drehzahl von “NNNN” pro
Minute | Para utilizar hasta un máximo de “NNNN” rpm | Voor gebruik
tot “NNNN” r.p.m | Op til “NNNN” o/m / Maksimi “NNNN” kierrosta
minuutissa | För användning upp till “NNNN” varv per minut | Para
uso até “NNNN” rpm | Για χρήση μέχρι και “NNNN” στροφές/λεπτό |
Produkt przeznaczony do stosowania przy prędkościach do “NNNN”
obr/min | K použití až do “NNNN” ot/min | max. “NNNN” rpm-ig
használható | “NNNN” devir/dakika’ya kadar kullanım için | Kan brukes
opptil “NNNN” omdreining per minutt.

DEVICE DESCRIPTION

Medtronic burs are interchangeable drill bits and burs intended for use with the Medtronic ENT and Xomed line of otologic drill handpieces. Burs are supplied in a variety of materials for improved cutting performance including: stainless steel with titanium nitride coating, carbide and stainless steel, tool-steel, carbide, and stainless steel with diamond studded heads.

OPERATOR NOTICE — The clinician is strongly recommended to have a thorough understanding of and experience with powered surgical equipment prior to use. Medtronic Xomed provides surgical technique booklets relating to the application of the various cutting blades and accessories for the XPS® 3000 System. The health care professional should seek out and review all other clinical reference materials as dictated by the individual patient's clinical condition.

INDICATIONS FOR USE

Medtronic Xomed burs are indicated for use in incising or removing bone and tissue during general otorhinolaryngology, head and neck, or otoneurological surgery.

CONTRAINDICATIONS

None are known.

WARNINGS

- DO NOT use burs above the speed indicated on the bur label. Exceeding speed limits may cause bur to break.
- Use adequate irrigation. The use of a bur without irrigation may cause an inordinate amount of heat buildup resulting in thermal injury to tissue.
- Always keep the cutting tip away from fingers and loose clothing.
- Operate the blade or bur only after the appropriate anatomical landmarks and the intended surgical site have been confirmed.
- Ensure bur is properly secured in the handpiece prior to surgeon activation.
- DO NOT change burs with handpiece running.
- Test for bur wobble (eccentricity) at desired speed prior to use. Use a bur guard, if needed. Select a new bur if wobble is observed or reduce speed.
- Excessive pressure applied to bur may cause bur fracture which may cause injury.
- DO NOT use dull, damaged, or bent burs. Change burs frequently for optimal performance. Use of dull burs can reduce handpiece cutting effectiveness and cause handpiece temperature to increase.
- DO NOT attempt to resharpen burs. Worn burs should be replaced with new ones frequently to ensure effective cutting and control of drill.
- Carefully inspect burs both prior to and following each use for signs of excessive wear, fragmentation, eccentricities or other defects. Replace any suspicious bur with a new one prior to use.
- Improper priming of Visao®/ Xcalibur® Hi-Speed Drill will result in excessive handpiece temperature.
- DO NOT modify accessories used with any handpiece.
- Bending or prying may break the blade or bur, causing harm to patient or staff.

SURGICAL PRECAUTIONS

- All reusable components of the device must be cleaned and sterilized and all disposable components replaced before using the device system on another patient.
- Use methods at the operative site (airway, throat, head, neck or sinus cavity) to control bleeding that could compromise patient safety during at-risk surgery.
- DO NOT use any parts other than Medtronic Xomed, Inc. components as damage or substandard operation could result.
- Excessive noise from the bur when drilling close to the cochlea or ossicular chain may cause hearing damage.
- The use of a bur without irrigation may cause an inordinate amount of heat build-up resulting in thermal injury to tissue.
- Excessive force may lead to bur bending, breakage or excess vibration. If the bur fractures during use, the resultant sharp surfaces may cause lacerations of the body tissue. A bur which fractures inside a patient may also cause tissue injury and necessitate emergency procedures for extraction.
- Eccentricity of the bur can cause bur vibration and may result in excess tissue and bone destruction and hearing damage.
- When operating or testing the Xcalibur®, Powerforma® and Visao® drill handpiece, ensure bur is properly inserted into the handpiece. Running the handpiece with an unsecured bur can damage the handpiece. Running a handpiece without a bur is not recommended.
- During procedures, operating room technicians and nurses should always check bur function, fit, tightness, and security in the handpiece before handing off to surgeon for use.
- **IMPORTANT:** Always examine operation of each bur in a handpiece before each use. Operating certain burs at high speeds can cause vibration of the bur. The following are suggestions for reducing bur vibration:
 - Reduce handpiece operating speed.
 - Use only burs that are rated for High Speed operation.
 - Use Xcalibur® Hi-Speed Extended 64 mm Bur Guards (REF # 3333645 or 3333655) with 64 mm burs in the Xcalibur® Hi-Speed Drill.
 - Use Xcalibur® Hi-Speed Extended 75 mm Bur Guards (REF # 3333625 or 3333635) with 75 mm burs in the Xcalibur® Hi-Speed Drill.
 - Use Xcalibur® Hi-Torque Extended Bur Guards (REF # 3333620 or 3333630) with 75 mm burs in the Xcalibur® Hi-Torque Drill.
 - Use Visao Hi-Speed Guards (REF # 3334625 or 3334635) with medium and long burs.
 - Operate handpiece at 50% speed when using burs 75 mm or longer.
 - Select a new bur.

INSTRUCTIONS FOR USE

Note: Medtronic Xomed burs are provided sterile for first time convenience. DO NOT use if package is opened or damaged. Broken sterile seal offers no protection against cross-contamination.

Before Surgery:

For Use with the Powerforma® handpieces: BUR ATTACHMENT

1. Using your thumb, pull back on the bur release button.
2. Insert desired Medtronic Xomed surgical bur into handpiece, giving bur a 1/4 turn as it is seated (or until it seats fully into the handpiece).
3. Allow the bur release button to slide forward.
4. Tug on bur to ensure bur is secured and locked into the handpiece.
5. **IMPORTANT:** Test drill by depressing foot pedal. The system is now ready to use.
6. To remove the bur, pull back on the bur release button, and pull the bur out of the chuck.

For use with the Xcalibur®/ Visao® handpieces: BUR ATTACHMENT

1. Using thumb and index finger, twist handpiece locking mechanism counter clockwise to unlock handpiece.
2. Insert desired Medtronic Xomed Surgical Bur into handpiece, giving bur a ¼ turn as it is seated (or until it seats fully into the handpiece).
3. Lock and secure bur by twisting handpiece locking mechanism clockwise.
4. Tug on bur to ensure bur is secured and locked into the handpiece.
5. **IMPORTANT:** Test drill by depressing foot pedal prior to use.
6. To remove the bur, twist the handpiece locking mechanism counter clockwise and pull the bur out of the chuck.

TROUBLESHOOTING

IF Burs wobble in Xcalibur®/Visao®/Powerforma® Handpieces...

- Reduce handpiece operating speeds.
- Use burs that are rated for the console speed selected.

- If necessary, use extended bur guards with Xcalibur®/ Visao® drills for 64 mm and 75 mm burs.
- Operate handpiece at 50% of full speed for 64 mm and 75 mm burs.
- Select a new bur.

CUSTOMER SERVICE INFORMATION

For further information regarding the use of this product or to report any problems, please contact Medtronic Xomed using the appropriate information provided on the blue and white contact information card packaged with each device; or contact your local distributor.

Please contact your Medtronic Customer Service Representative for Warranty information for this product.

CLEANING AND STERILIZATION GUIDELINES

Warnings / Precautions	Remove burs from the handpiece before cleaning and sterilizing. Before sterilization, carefully inspect the bur tips, bur flutes, under a microscope or magnifying glass for any irregularities or eccentricities. Discard any burs that show signs of damage or wear. Cold soak in glutaraldehyde, chlorine, or ammonium solutions, or dry heat sterilization is not recommended as damage to the bur may occur.		
Limitations	Discard any burs that show signs of damage or wear.		
Instructions			
Point of Use	Remove burs from the handpiece before cleaning and sterilizing. Promptly and thoroughly rinse instruments with deionized water after each use.		
Containment and Transportation		No particular requirements	
Preparation for decontamination	Promptly and thoroughly rinse instruments with deionized water after each use.		
Cleaning: Automated (Do NOT use ultrasonic washer)	Remove instruments and equipment from any sterilization trays before placing into washer baskets. Orient devices following recommendations of washer/disinfector manufacturers. Use alkaline or neutral pH detergent recommended by washer/disinfector or detergent manufacturers. These products have been validated for effective cleaning using an automatic washer/disinfector cycle consisting of a minimum 44 minutes total time, including a pre-wash, main wash & rinse, and thermal rinse. The thermal rinse shall be at least 10 minutes long at a minimum temperature of 60°C.		
Cleaning: Manual	Soak in lukewarm*, mild* enzymatic detergent, and deionized water for a minimum of two minutes. Then clean ultrasonically in lukewarm* solution of mild* detergent and deionized water for at least 30 seconds. Rinse thoroughly with deionized water and wipe dry. *less than 43°C; pH 7.0 - 8.5 Note: When using an ultrasonic cleaner or a spray washing machine, follow the manufacturer's recommendations, particularly with regard to articulated instruments and positioning of instruments.		
Disinfection	No particular requirements		
Packaging	No particular requirements		
Sterilization (Temperatures are minimum required, times are minimum required)	Cycle:	Gravity	Gravity
	Temperature:	121-123°C	132°C
	Time:	30 min	10 min
	Cycle:	Pre-Vac:	Pre-Vac: (FR/WHO)
Temperature:	132°C	134°C	134°C
Time:	4 min	18 min	3 min
Drying: 8 minutes, or until visibly dry			
STERRAD Sterilization: Not validated			
100% EtO: Not validated			
Maintenance, Inspection and Testing		Discard any burs that show signs of damage or wear.	
Storage	Store in a clean dry place.		
Additional Information	None		

Note: The instructions provided above have been validated by the manufacturer as being CAPABLE of preparing the product for re-use. It remains the responsibility of the processor to ensure that the reprocessing as actually performed using equipment, materials and personnel in the reprocessing facility achieve the desired result. This normally requires validation and routine monitoring of the process.

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Jacksonville, FL 32216 U.S.A.