

# **Disposable biopsy forceps**

## **Instruction**

**【Product name】** Disposable biopsy forceps

**【Main structure of product and its composition】** Disposable biopsy forcep is composed of tong head, tube, slip ring, handle and needle, etc. Pliers head, tube and needle made, 06 cr19ni10 slip ring and the handle is made of ABS, which adopts PE plastic coated layer. According to the structure is divided into A, B, C, D, E, F, G, H eight, A for the flat without needles, no plastic bag. B said the flat with the needle, not plastic package. C said the flat needle, plastic coated. D said the flat belt needle and plastic coated. E said tooth mouth without needles, no plastic bag. F said tooth mouth with needle, not plastic package. G the tooth mouth no needle, plastic coated. H said tooth mouth with needle, plastic coated. Tongs closed head diameter is 1.8 mm and 2.3 mm. Products should be sterile.

**【Specification and type】** ATE-QYQ-A(B、C、D、E、F、G、H)-18(23)X500(800、1050、1200、1500、1600、1800、2300、2600)

**【Usage】** Used with endoscope, for endoscope gastric and duodenal biopsy samples.

### **【Method of use】**

Apart single packing, will head and outer tube into endoscopic cutters and aim the tong head lesion group, promote the slip ring open clamp head, near by sampling organization, then pull the slip ring closed jaw, pull back the tube out biopsy forceps.

### **【Matters needing attention】**

1. The operator of the use of this product must be a medical personnel, and must have the corresponding qualification;
2. This product by the EO sterilization, sterile products, valid for two years after the

sterilization of qualified, recognizing when using single packing pincer-like device failure, overdue it is prohibited to use;

3. The single packing damage, spare parts, packaging banning the use of foreign body in it, and destroy;

4. This product is only for one-time use, destroyed after use;

**【Contraindications】**













- 1, acute perforation of esophagus, stomach and duodenum.
- 2, serious heart, lung, kidney, brain dysfunction and multiple organ failure.
- 3, psychosis, and impaired consciousness obviously can't collaborator.
- 4, and other specific by professional doctor according to the actual condition of the patient.

**【Packaging and storage】**

1.Single packaging uses paper-plastic package; middle packaging uses cartons; external packaging uses corrugated carton.

2.Products after packing should be stored in a well-ventilated room with relative humidity not more than 80% and no corrosive gas.

**【Symbol Explanation】**

No	Babel/symbol	Instruction	No	Babel/symbol	Instruction
1		Do not use if the package damaged	2		Single use
3		Manufacturer	4		Production date
5		Lot Number	6		Expiry Date
7		This product is sterile, adopting the way of EO sterilization	8		EU representative
9		See the instruction for other details	10		Humidity limit
11		Avoid the sun	12		Avoid rain

【Shelf life】 2 years




Jiangsu Ate Medical Technology Co., LTD

No. 8 Lanxiang Road, Wujin Economic Development Zone, Jiangsu, China

【Tel】 0519-85288988

【Fax】 0519-85288060

【Zip code】 213149

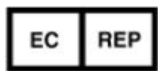

ZOUSTECH S.L.

Pso. Castellana, 141- Planta 19, 28046- Madrid, Spain

【Tel】 +34 694426446

【Fax】 +34 917915466

【Zip Code】 28046



【Version NO.】 A/1

【Email】 sales@ate-medical.com

【Revision date】 20th Dec 2020