

## The STEMA Trocars

- EJ - 001 - XXX
- EJ - 011 - XXX



**Model: EJ - 011 - XXX**



**Model: EJ - 001 - XXX**

### Cleaning/Sterillization

Immediately after use the instruments have to be soaked in a combined disinfection and cleaning solution in order to prevent operation residues to dry on and a transmission of germs in the endoscopy rooms. Cleaning agents and disinfection solution have to match. Please observe the respective guidelines for these substances. The instruments have to undergo a thorough cleaning process, because detergents, blood,

pus, and protein residues may influence the disinfection and sterilization procedure. After the cleaning procedure, all moving parts have to be oiled. All revolving stop cocks and pistons have to be lubricated.

Proper care and service are necessary for the safe use of medical products. We therefore recommend a thorough check of function, completeness and damages (especially porous and cracked rubber and plastic parts), sharp edges and rough surfaces.



CE 0483



### STEMA Trocars

#### Intended purpose

Reusable surgically invasive device for temporary use.  
It enables access to the operation site

#### Indication

This instrument is designed for use in minimally invasive surgery and, in particular, laparoscopy. The sheath serves to introduce instruments, telescopes and liquids into the surgical site.

#### Contraindication

This instrument is not designed for use on the central nervous and circulatory system.

### STEMA Trocars



***Please pay attention to our general information for Care, Maintenance and Cleaning***