

# Shoulder Area

30 mm.

5 mm.

### Indications:

To cleanse minor wounds and piercings.

### Directions for Use:

- Before use, spray a short stream of NeilCleanse® Piercing Aftercare Saline Spray in the air or on intact skin to test stream and to clear nozzle
- To dispense, aim nozzle at the affected area or piercing and spray from a distance of 2 to 4 inches, or as is comfortable
- Cover with clear cap after each use

### Ingredients:

USP Grade Water, USP Sodium Chloride (9 mg/mL)  
Preservative Free

### Recommended storage conditions:

- Store between 59°F and 86°F (15°C to 30°C). Product can temporarily be exposed to temperatures ranging from 41°F to 131°F (5°C to 55°C) during transportation.
- Store out of sunlight, away from heat sources and avoid freezing

MDSAP/ISO13485:2016,  
Certified Company

©NeilMed® 2000-2022

# NeilMed®

  


## neilcleanse®

# Piercing Aftercare

## FINE MIST

WOUND WASH  
Sterile Isotonic Saline Solution  
NET WT 6.3 oz (177.5 g)

5 mm.

### WARNINGS:

- Do not use if security sleeve is missing or tampered with
- Do not use on burn injuries. Do not use in body orifices, ear canals or eyes.
- Not for injection
- For individual use only
- Apply as needed. Consult a healthcare professional for continued wound care management and for use on children younger than 2 years of age
- Contents under pressure; please do not puncture or incinerate
- Keep out of reach of children
- Do not use if package is damaged
- If redness, swelling, or pain worsens, please seek the advice of your health care practitioner

NPN 80032448

**NeilMed® Pharmaceuticals, Inc**  
601 Aviation Blvd., Santa Rosa, CA 95403 USA  
Tel: 1 (877) 477-8633 • www.neilmed.com

Lot / Expiration Date on the bottom of the canister



MDSS GmbH Schiffgraben 41  
30175 Hannover, Germany



112 mm.

5 mm.

185,35 mm.