


## Introduction For Use Guedel Airway

### Intend for use

For Airway management of patient. Guedel Airway is used for supporting the other devices throughout the patient throat to maintain the airway and to prevent the patient from biting the tongue during airway control during anesthesia, surgery, resuscitation or postoperative.

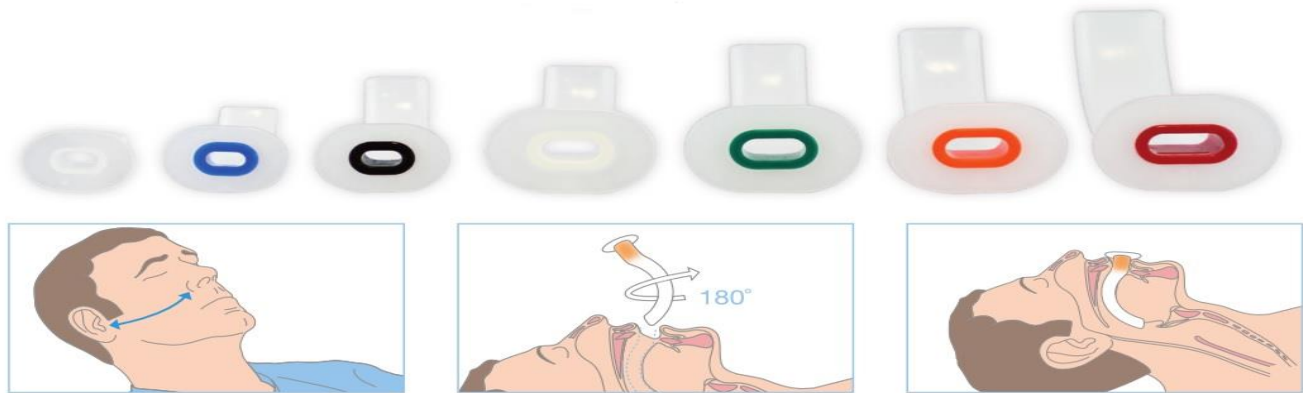

### Introduction for use:

1. Check the size of the Airway through color coding on the product
2. Place the patient on his or her back for about 45°, carefully check that the patient's mouth and airway are clear or not.
3. Tear off the airway packaging and check the integrity and size of the product again through color coding on the airway mouth
4. Lubricate the airway with water-soluble lubricant.
5. One hand lift the patient's head and insert with the curved surface opposite to the tracheal curve and the airway direction then rotate the airway 180 degrees following the tracheal curve as shown above.
6. Slowly insert the airway until the top rim of the airway touches the patient's lips, stop.
7. Insert necessary equipment into the airway to control the patient's airway.

 Flexicare Medical Limited.  
Cynon Valley Business Park  
Mountain Ash  
Mid Glamorgan  
CF45 4ER, UK.

+44 (0)1443 474647  
+44 (0)1443 474222  
enquiries@flexicare.com  
www.flexicare.com



FC1603  
02/2012  
VP5400-06