

INSTRUCTIONS FOR USE

Iris Retractor - Disposable

DESCRIPTION

The Iris Retractor is made of bright blue polypropylene fiber and has an adjustable silicone stopper for temporary iris fixation during surgery. In cases where iris dilatation cannot be achieved pharmacologically, the Iris Retractor is a simple, fast and safe alternative. The Iris Retractor gently stretches and retains the iris for maximum visibility during the surgical procedure.

INTENDED USE

The Iris Retractor is used to manually dilate the iris in small pupils during anterior and posterior segment surgeries.

CAUTIONS

- (USA) Federal law restricts this device to sale by or on the order of a licensed physician.

PRECAUTIONS

- Do not use if package has been opened or damaged.
- Do not use if expiry date shown on the label is exceeded.
- This product is for single use. Do not reuse. Do not re-sterilize.
- Reuse and/or re-sterilization may compromise device performance, which could cause serious harm to the patient's health and safety.

INSTRUCTIONS FOR USE

PACKAGING REMOVAL:

- Peel pouch and remove blister container that holds Iris Retractors.
- Place blister on the palm of your hand
- Slide your thumb between the top and the bottom of the blister to open

INSERTION:

- The cornea must be clear to control positioning of the retractors.
- Make a self-sealing stab incision at the limbus using 0.5mm wide spatula shaped knife.
- Hold the Retractor with a small needle holder or forceps near the hooked end and introduce the retractor into the eye through the incision.
- Turn the needle holder so that the retractor can be hooked to the edge of the iris and retract as much as necessary toward the entry site. Adjust the retractor to the appropriate position and tension by moving the silicone stopper towards the cornea.

REMOVAL:

- Slide off the silicone stopper to release the iris.
- Unhook the Retractor from the iris.
- Slowly withdraw the Retractor. The hooked end will straighten and slip out of the self-sealing incision.

