


# Gibeck Humid-Vent® Filter Bacterial/Viral Filter and Heat and Moisture Exchangers

Instructions for use

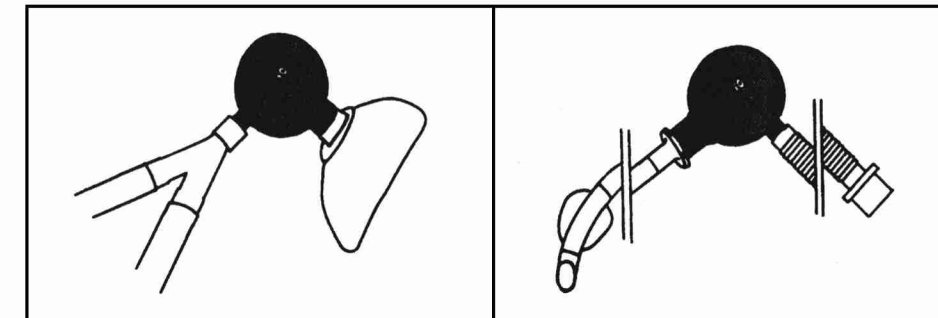

Teleflex Medical Sdn. Bhd.  
Lot PT 2577, Jalan Perusahaan 4,  
34600 Kamunting, Perak, Malaysia  
Made in Malaysia

EC REP

Teleflex Medical  
IDA Business & Technology Park,  
Dublin Road, Athlone,  
Co. Westmeath, Ireland

CE  
0124

en

Humid-Vent Filters (HMEF) are combined Heat and Moisture Exchanger (HME) and Bacterial/Viral Filter for humidification and bacterial/viral filtration during anaesthesia and ventilator care. Expert clinical judgement must be used in assessing the patients humidification requirements.

- Position patient side connector of the HMEF higher than ET connector to avoid accumulation of condensation.
- The tubing should be lower than the HMEF.
- If patient is re-positioned it may be necessary to reposition the Humid-Vent Filter.



**Caution:**

- Do not use if package is broken or product is damaged.
- Do not use with explosive anaesthetic gases.
- Replace every 24 hours or as needed to prevent accumulation of secretion.
- Replace unit immediately if soiled with secretion or otherwise obstructed.
- Humid-Vent Filters are designed for single use only and should not be cleaned and re-used.



- Re-processing of medical devices intended for single use only may result in degraded performance or a loss of functionality. Re-use of single use only medical devices may result in exposure to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilization methods and instructions for reprocessing to original specifications are not available for these medical devices. This product is not designed to be cleaned, disinfected, or sterilized.
- Take total system dead space into consideration before use.
- US federal law restricts this device to sale by or on the order of a physician.
- This product does not contain natural rubber latex.

**Storage Instruction:**

Keep away from sunlight and keep dry. Do not use if the product sterilisation barrier or its packaging is compromised.

**Contraindications:**

- Humid-Vent Filters should not be used with patients producing fulminating frothy secretions within their airways, or patients with haemoptysis.
- Do not use in conjunction with conventional humidifiers and nebulizers.
- Do not add moisture to the HMEF.

**Explanation of important symbols and markings on the product label.**

Qty	Quantity, Number of items
	Do not use if package is damaged



*Phạm Thị Thu Hằng*  
GIÁM ĐỐC

STERILE EO Rx Only