# HbA1c 2R CONTROL LOW

Lot: XXXXXXX

Cat. No.	Pack Name	Packaging (Content)
XSYS0098	HbA1c 2R CON L	4 × 0.5 ml

EN

## INTENDED USE

HbA1c 2R CON L is a set of four lyophilized controls based on human blood material (erythrocytes). HbA1c concentration in HbA1c 2R CON L is normal.

## STORAGE

The controls both unopened and opened must be stored at 2–8 °C, protected from light and heat.

## STABILITY

Unopened controls are stable until the date of expiry stated at the label.

After opening and reconstitution are controls stable for one week if stored under 2–8 °C and if contamination and evaporation are avoided after having opened the vials.

Proper storage and handling must be observed.

## WARNINGS AND PRECAUTIONS

Reagents of the kit are not classified like dangerous.

Each individual blood donation used for production of HbA1c 2R CON L was found to be non-reactive when tested with approved methods for HBs-Ag, HIV-Ag/Ab and HCV-Ab HTLV-1-Ab. As there is no possibility to exclude definitely that products derived from human blood transmit infectious agents, it is recommended to handle the controls with the same precautions used for patient specimens.

## PREPARATION

Before use perform reconstitution of control by adding 500 µl of hemolyzing solution (R3) into the vial, mix gently and allow to stand for 5 minutes. After reconstitution is control ready to use. Don't prepare hemolysate.

## ASSIGNED VALUE

Target values according to IFCC (mmol/mol):

	Lot No.	Expiry Date	Targer value (mmol/mol)	Range (mmol/mol)	1 SD (mmol/mol)
HbA1c Control Low	xxxxxxx	mm/yyyy	x.xx	x.xx–x.xx	x.xx

Target values according to DCCT/NGSP (% NGSP):

	Lot No.	Expiry Date	Targer value (%)	Range (%)	1 SD (%)
HbA1c Control Low	xxxxxxx	mm/yyyy	x.xx	x.xx–x.xx	x.xx

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