

STERIS®



**PROLYSTICA® 2X CONCENTRATE
ALKALINE DETERGENT
DIRECTIONS FOR USE**

DIRECTIONS FOR USE

Automated Applications

Prolystica 2X Concentrate Alkaline Detergent may be automatically dispensed at 1/8 - 1/2 oz./gal. (1 - 4 mL/L) depending on water quality and soil load. For use on stainless-steel surgical instruments, select soft metals and apparatus where blood, protein, carbohydrates, mucus and lipids form hard to remove soils.

Manual/Ultrasonic Applications

Fill sink with warm water to the appropriate level to fully immerse surgical instruments. Dilute product at 1/8 - 1/2 oz./gal. (1 - 4 mL/L) of warm water. Soak for a minimum of one to five minutes. Dried-on proteinaceous soil may require longer soak times. After cleaning, all surfaces should be thoroughly rinsed with warm water. Solutions should be discarded daily or when visibly soiled.

This is a complete product. **Do not add other chemicals** such as bleach or detergents. Added chemicals may destroy the effectiveness of the product.

Consult the medical device manufacturer's recommended reprocessing instructions to determine any additional procedure necessary to clean their particular medical device.

Medical devices should be sterilized or disinfected by appropriate means in a separate subsequent step.

STORAGE AND DISPOSAL

Store in a cool place, below 32°C (90°F). Keep out of the reach of children. Dispose in accordance with local, state and federal regulations. Empty containers should be rinsed and discarded in a waste container or offered for recycling, if possible. Do not use empty containers for other purposes.

SERVICE

Sales

Service is one of the most important ways to verify consistent quality of the facility's performance and operation. A tailored service program by STERIS provides effective, trouble-free operations.

Technical

Contact STERIS for information regarding product applications, the product in general or copies of Safety Data Sheet (SDS). For information regarding human and environmental exposure, call STERIS's St. Louis Operations Regulatory Affairs Group at 314-535-1395.

For further information, please contact:

STERIS®



STERIS Corporation
5960 Heisley Road
Mentor, OH 44060-1834 • USA
440-354-2600 • 800-548-4873
www.steris.com