

STERIS®



**PROLYSTICA® 2X CONCENTRATE
ENZYMATIC PRESOAK AND CLEANER
DIRECTIONS FOR USE**

DIRECTIONS FOR USE

Automatic Washer/Disinfectors:

Prolystica 2X Concentrate Enzymatic Presoak and Cleaner may be automatically fed at 1/8 to 1/2 fl. oz/gal (1 to 4 mL/L).

Manual/Ultrasonic Applications:

Dilute 1/8 to 1/2 fl. oz/gal (1 - 4 mL/L) in warm water. Activity increases as the water temperature increases. Clean a minimum of one to five minutes. Cleaning time may be longer with dried on proteinaceous materials or if lower temperatures are used. Do not exceed 130°F (55°C). After cleaning, rinse thoroughly or transfer to next cleaning operation. Discard used solutions daily or when visibly soiled.

This is a complete product. Do not add other chemicals, such as bleach or detergents. Added chemicals may destroy the effectiveness of this product.

Medical devices should be sterilized or disinfected by appropriate means in a separate subsequent step.

STORAGE AND DISPOSAL

Store in a cool place (below 90°F/32°C). Keep out of the reach of children. Dispose in accordance with local, state and federal regulations. Empty containers should be rinsed and discarded in a waste container or offered for recycling if possible. Do not use empty containers for other purposes.

SERVICE

Sales

Service is one of the most important ways to verify consistent quality of the facility's performance and operation. A tailored service program by STERIS provides effective, trouble-free operations.

Technical

Contact STERIS for information regarding product applications, the product in general or copies of Safety Data Sheets (SDS). For additional information, call STERIS's St. Louis operations regulatory affairs group at 314-535-1395.

For further information, please contact:

STERIS®



STERIS Corporation
5960 Heisley Road
Mentor, OH 44060-1834 • USA
440-354-2600 • 800-548-4873
www.steris.com