



**MEDIFIRST Co., Ltd.**

#1049-16, Charyeonggogae-ro, Gwangdeok-myeon, Dongnam-gu, Cheonan-si, Chungcheongnam-do, Korea  
Tel: +82-41-522-2650 Fax: +82-41-522-2673 <http://www.medifirst.co.kr>

## **INSTRUCTIONS FOR USE**

### **Infusion Set**



## **PRECAUTIONS**

- Do not use it the unit package or the product has been damaged or soiled
- Use immediately after opening the unit packaging.
- Do not use if protective end caps are not in place.
- Dispose of safely after single use to avoid risk of infection.
- For single use only. Do not reuse. Do not resterilize. Do not reprocess.

## **INSTRUCTIONS FOR USE**

- Close the roller clamp
- Insert the needle into the infusion bottle
- Hang the infusion bottle on the rack, adjust the amount of liquid to full ½ the body of the drip chamber
- Open the ventilation button and roller clamp, expel all the air in the tube
- Plug the infusion set into the needle, adjust the drip rate accordingly and start the infusion.