**ASSEL S.r.l.**  
Via E. Barsanti 13/A – 00012 Guidonia (Rm)  
Tel.: +39 0774 357492  
Fax: +39 0774 372179  
e-mail: info@asselitaly.eu

AZIENDA CERTIFICATA  
UNI EN ISO 9001:2008  
UNI EN ISO 13485:2012  
CERTIFIED COMPANY

## SYSTEM CONTROL P

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6 x 5 ml

### • Characteristics

System Control P is a human lyophilised serum. It is intended for control of accuracy of clinical chemistry methods with manual and automated procedures.

Constituents concentrations and activities are in the normal range

### • Reagents

Lyophilized human serum.  
Biological additives and bacteriostatic agents.  
The concentration and activities values of the components are indicated in the attached value sheet and are lot-specific.

### • Preparation

- Open the vial avoiding any loss of material.
  - Reconstitute with 5.0 ml exact of distilled water.
  - Cap the vial, mix thoroughly to dissolve the content avoiding forming foam.
  - Bring to room temperature for about 30 minutes before use.
- Inaccurate reconstitution and/or improper storage can cause erroneous results.

### • Stability and Storage of Reagents

The Control serum is stable until the expiration date stated on the label when stored tightly closed at 2-8 °C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth.

Store the control tightly capped when not use.

- After reconstitution is stable for:
  - 12 hours at 15-25 °C.
  - 5 days at 2-8 °C.
  - 1 month at -20 °C.
- Bilirubin (stored protected from light):
  - 2 hours at 15-25 °C.
  - 6 hours at 2-8 °C.
  - 2 weeks at -20 °C

#### **Note:**

1. Avoid freeze-thawing cycles.
2. It is suggested to subdivide reconstituted control into aliquots and store at -20 °C.

### • Precautions and warning

Whenever agents infettantis, chemical reagents, reagents of human/animal origin, blood or other biological liquids are manipulated, it is advisable to follow the most common recommendations and take all the necessary hygienic precautions as the monouse gloves.

#### **BIOLOGICAL RISK**

\* Each unit of source material used in the preparation of Control has been tested by an approved method and found non – reactive for HBsAg and negative for antibodies to HCV and HIV 1/2.

However, no known test method can offer complete assurance that products derived from human blood will not transmit hepatitis, AIDS or other infectious diseases.

This product, like all materials of human origin, should be handled as potentially infectious biological material.

### • Waste disposal

Please consult local regulations for a correct waste disposal.