

SRI TRANG GLOVES (THAILAND) PUBLIC COMPANY LIMITED

10 Soi 10 Phetkasem Road, Hat Yai, Hat Yai, Songkhla 90110 Thailand
Tel. (66) 074-344-663 Fax. (66) 074-344-677

SCT.PD.SP.05.059-010222 R.04

DAR-SCT-2022-0012

PRODUCT SPECIFICATION SHEET : NCCOGF-E-OS-B-NS

PAGE 1 of 1

GLOVE SPECIFICATION

Product Specification Group Code :	NCCOGF-E-OS-B-NS
Product Code :	DNCFEOG (With Color)
Product name :	Nitrile Examination Gloves, Powder Free, Polymer Coated, Non Sterile
Grade :	OS Spec, Medical Grade

GENERAL INFORMATION

Type :	Single-use disposable examination or protective glove
Marking :	Gloves are not marked to designated size
Shape :	Ambidextrous
Material :	Nitrile Butadiene Rubber
Cuff :	Rolled cuff
Surface :	Textured at Finger Tip

Thickness : (single wall according to ASTM D6319, ASTM D3767)	Finger : Min. 0.05mm	Palm : Min. 0.05mm
---	-----------------------------	---------------------------

Residual powder : (according to ASTM D6319, ASTM D6124)	Max. 2.0mg/glove	Tensile Strength : (according to ASTM D6319, ASTM D412, ASTM D573)	Before Aging : Min. 14MPa After Aging : Min. 14MPa
---	------------------	--	---

Length : (according to ASTM D6319, ASTM D3767)	XS, S : Min. 220mm M, L, XL : Min. 230mm	% Elongation : (according to ASTM D6319, ASTM D412, ASTM D573)	Before Aging : Min. 500% After Aging : Min. 400%
--	---	--	---

Available Sizes & Palm width (mm) : XS : 70+/-10, S : 80+/-10, M : 95 +/-10, L : 110+/-10, XL : 120+/-10 (according to ASTM D6319, ASTM D3767)
--

Storage: 3 years storage life (date of expiration see on the box). Keep area cool, dry and dust free, avoid ventilation and storage close to photocopy equipment. Copper-ions discolor the glove. Protect gloves against ultraviolet light sources, as sunlight and oxidizing agents. Storage above 30°C will lead to accelerated aging and should be avoided.

STANDARDS, GUIDELINES AND QUALITY CERTIFICATION

(Every mentioned standard is used in the latest edition.)

Certification : ISO 9001, ISO 13485

Conformity to standards : ASTM D6319

Please report any incidents with the products to STGT_TA@srtranggroup.com and regulatory@srtranggroup.com without undue delay. Our Product Safety Officers will review all information provided and might contact the original informant for further clarification.

SRI TRANG GLOVES (THAILAND) PUBLIC COMPANY LIMITED

10 Soi 10 Phetkasem Road, Hat Yai, Hat Yai, Songkhla 90110 Thailand
Tel. (66) 074-344-663 Fax. (66) 074-344-677

SCT.PD.SP.05.059-010222 R.04

PRODUCT SPECIFICATION SHEET : NCCOGF-E-OS-B-NS

DAR-SCT-2022-0012

Appendix of specification

APPENDIX : 12

Product Specification Code : NCCOGF-E-WHT-OS-BD-NS
Product Code : DNCFEOG WHITE
Material : Nitrile Butadiene Rubber
Treatment and Feature : Powder Free, Polymer Coated, Offline Chlorinated

SAMPLING, INSPECTION AND FINAL RELEASE INFORMATION

Product Specification Code	Glove color	Characteristics			Test-Method
		Barrier Defects*	Major Visual Defects*	Minor Defects*	
NCCOGF-E-WHT-OS-BD-NS	WHITE	AQL ≤ 2.5	AQL ≤ 2.5	AQL ≤ 4.0	ASTM D5151

Barrier Defects :

Inspection level G I (ISO 2859-1)

Holes that results in the malfunction of glove as a protection barrier either physically or not physically affect the appearance.

Major Visual Defects :

Inspection level G I (ISO 2859-1)

Defined defect that disrupt the glove's function or defect that cause an uncomfortable feeling to the user.

Minor Defects :

Inspection level G I (ISO 2859-1)

Visual defects aggregated are non-conformities of lower degree of concern, which do not prevent correct use of gloves. Minor cosmetic defect which does not affect on the glove's performance

Remark :

* Defect Characteristics following In-house practice.