

Blood LANCETS

CE 0197

Instruction For Use

SPECIFICATION

BL-21G, BL-23G, BL-26G, BL-28G, BL-30G, BL-32G, BL-33G.

INDICATION

It is intended to be used by a healthcare provider or patient self to manually puncture the skin of a patient to obtain a small blood specimen.

It can be used with a lancing device.

INSTRUCTION FOR USE

Method 1: Twist the protective cap → prick skin → recapping → discard into sharp container.

Method 2: You can insert a lancet to be able to obtain blood with the lancing device.

- Remove the cap from the lancing device;
- Insert a new lancet into the lancet holder as far as it will go. The lancet must audibly click into place.
- Twist the protective cap off of the lancet.

Note: Consult the IFU of lancing device (provided by other supplier) prior to use.

COMPATIBLE WITH LANCING DEVICES:



CAUTION

- Do not re-use;
- Do not re-sterilize;
- Do not use in case of loose or detached sterile cap;
- Discard the product after using.

PRECAUTION

Do not use the lancet if the safety tab is missing or loose when you take the lancet out of bag. Use caution whenever the lancet needle is exposed.

WARNING

- Each lancet may only be used once to obtain blood.

- Used lancets must be safely discarded after a single use.
- Improper use of blood lancets can increase the risk of inadvertent transmission of blood borne pathogens, particularly in settings where multiple patients are tested.










SHELF LIFE

5 years.

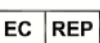PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.


EXPLANATION OF SYMBOLS USED ON LABELS AND INSTRUCTIONS FOR USE

	Manufacturer		Sterilized using irradiation
	Authorized representative in the European Community		Do not re-sterilize
	Use-by date		Do not re-use
	Batch code		Consult instructions for use
	Caution		



Promisedmed Hangzhou Meditech Co., Ltd.

Address: No. 1388 Cangxing Street, Cangqian Community, Yuhang District, Hangzhou City, 311121 Zhejiang, China.

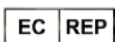

OBELIS S.A

Address: Bd. Général Wahis, 531030 Brussels, Belgium.

Issued date:2020-05-04 Ver. A/0