

# POLYBOND INDIA PVT LTD MEDICAL DIVISION

Artwork No.:-

MQA-IS-76, REV NO. 10, DATE 16/06/2020

## INSTRUCTION SHEET OF B.T SET VENTED

### INSTRUCTIONS FOR USE

B.T Set (ECOTRAN - V)

#### MATERIAL USED:

- Polypropylene, Acrylonitrile Butadiene Styrene (ABS), PVC, Latex Rubber, SS Cannula, Nylon.

#### INTENDED USE :

- BT set are used to administer blood in to human circulating system
- By using intravenous catheter and cannula.

#### INDICATIONS :

- Infusion of Blood and Blood component.

#### CONTRAINDICATIONS :

- Not to be used in patients with known hypersensitivity to any material of construction.
- with the exception of 0.9% saline, no medications or solutions should be added to whole blood or any blood component.

#### INSTRUCTION FOR USE :

- Close the roller clamp ( Flow regulator).
- Remove the protective cap from the spike & insert the spike into the container to its full length.
- Squeeze the drip chamber & adjust the fluid.
- Remove the protective cap of IV needle & open roller clamp (flow regulator).
- Expel the air from the PVC tube & close roller clamp.
- Insert the needle in the vein & regulate the flow by adjusting roller clamp to achieve desired flow rate.

#### DURATION OF USE:

- Recommended maximum duration of use is 4 hour or 4 units, whichever come first, or between Blood and Blood products.

#### ⚠ CAUTION:

- The use of this product is restricted to a qualified doctor or a paramedic staff.
- Read instructions before use.
- The product should be used according to the instructions for use.
- POLYBOND DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.
- Visually inspect and carefully check the product and packing before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- Do not expose to extreme temperature, humidity and direct sunlight as it can affect packing / sterility integrity.
- The product is guaranteed non-toxic, sterile and non-pyrogenic if the package is not opened or damaged.
- The product should be used immediately after package is opened.
- For single use only. Do not reuse, reprocess or re-sterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/ or lead to device failure which, in turn, may result in patient injury and illness.
- Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another.
- Contamination of the device may lead to infection or illness of the patient.
- After using please dispose of the product by placing the used product back into the opened packing and dispose the pack safely and responsibly.
- Ensure safe and proper disposal of used/discarded product and packaging to avoid adverse effect to environment, children and stray animals.
- The disposal should be in accordance to local laws in the country of use.
- The device should not be use in the children , pregnant women, Lactating mothers & Nursing women.
- The device contains latex which may allergic like skin,eye,nose irritaion lining of the stomach to patient,user,pregnant women,newly born baby & lactating mothers.
- Check expiry date prior to use.



Manufactured By:

Polybond India Pvt. Ltd.  
Sanaswadi, Pune 412208,  
Maharashtra, INDIA

Customer care No.: +91-2138679313

Email: [eversure@rathigroup.com](mailto:eversure@rathigroup.com)

Web: [eversuremedical.com](http://eversuremedical.com)



CMC Medical Devices & Drugs S.L.  
C/ Horacio Lengo No 18 CP 29006, Málaga - SPAIN  
Ph +34951214054  
Email: [info@cmcmedicaldevices.com](mailto:info@cmcmedicaldevices.com)



Caution



Lot No/ Batch code



Date of Manufacturing



Use by Date



Do Not Resterilize



Single Use



Sterilized by EO



Fragile Handle With Care



Keep Away from Direct Sunlight And Radiation



Temperature Limitation



Keep Dry



CE 2460



Pyrogen Free



PHT DEHP



LATEX



Do not use if package is damaged.



REF



Catalogue Number



Manufacturer



EC REP



Authorized representative in the European Community



Consult instruction for use

200.00 mm

140.00 mm

White Maplitho Paper, 70 GSM, Printing Colour - 100% Black, Size - 140mm X 200mm

Plant Head

Marketing Head

Managing Director