


# Headrests/ Gối cổ định bệnh nhân

## USER MANUAL

### HƯỚNG DẪN SỬ DỤNG



PHÓ TỔNG GIÁM ĐỐC  
*Đoàn Anh Tuấn*

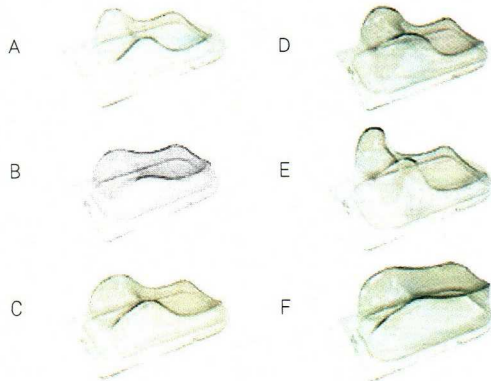

- Select the type of headrests according to the treatment indication
- Place the headrests on the top of the couch, at the marked position
- Fix the head of patient as the treatment indication. Place lowest elevation of the headrest at superior end of table allowing curved surface to properly position the neck
- User must clean device between uses according to instruction and follow infection control policies established by your facility for avoiding cross-contamination

• *Chọn loại gối theo chỉ định điều trị*

• *Đặt gối lên trên bàn bệnh nhân theo vị trí đã được đánh dấu*

• *Cố định đầu bệnh nhân theo chỉ định điều trị. Đặt phần thấp nhất của gối tại đầu bàn điều trị để đảm bảo độ cong thích hợp cho vị trí cổ bệnh nhân*

• *Người sử dụng phải vệ sinh các thiết bị giữa các lần điều trị và tuân thủ các quy tắc về kiểm soát nhiễm khuẩn để tránh việc lây nhiễm chéo*