


TP.HCM, ngày 01 tháng 07 năm 2022

MẪU NHÃN GÓC



ENTERAL FEEDING BAG SET
DOUBLE: 1200ML

PATIENT:.....
NUTRIENT:.....
ROOM:..... DATE:.....

DIRECTIONS:


- 1-CLOSE CONTROL CLAMP.
- 2- FILL CONTAINER WITH DESIRED AMOUNT OF ENTERAL FORMULA AND CLOSE BAG.
- 3- HANG BAG ON IV POLE WITH BUILD IN HANGER ON BAG.
- 4-ADD ICE TO POUCH,IF DESIRED.
- 5- SLOWLY OPEN CONTROL CLAMP AND FILL ENTIRE LINE WITH FLUID. (AVOID FILLING DRIP CHAMBER COMPLETELY.)
- 6- CLOSE CONTROL CLAMP.
- 7- ATTACH ADAPTER TO FEEDING TUBE. SLOWLY OPEN CLAMP TO ESTABLISH DESIRED DRIP RATE. BEGIN FEEDING.

(1ml= 20 drops approximate)

WARNING:
THIS SET IS INTENDED FOR ENTERAL FEEDING ONLY NOT FOR IV USE.

PRECAUTIONS:
* FOR SINGLE PATIENT USE ONLY.
* IT IS RECOMMENDED THAT THIS DEVICE BE REPLACED EVERY 24 HOURS.

Approximate Volume in milliliters



1200
1100
1000
900
800
700
600
500
400
300
200
100
50

SUZHOU YUDU MEDICAL CO., LTD
EASTERN XINZHUANG (ZHANGQIAO), TOWN
21552 CHANGSHU, CHINA

MADE IN CHINA





ENTERAL FEEDING BAG SET

DOUBLE: 1200ML

CAUTION :

Store in cool dry place and keep out of sunlight.

For single patient use only.

Replace this device every 24 hours.

Contents sterile unless package is opened or damaged.

STERILE EO



LOT



 SUZHOU YUDU MEDICAL CO., LTD
EASTERN XINZHUANG(ZHANGQIAO), TOWN
21552 CHANGSHU, CHINA

MADE IN CHINA

OPEN HERE



MẪU NHÃN PHỤ

1. Tên trang thiết bị y tế : Túi cho ăn loại đôi
2. Chủng loại thiết bị y tế : theo phụ lục
3. Quy cách đóng gói :
4. Số lưu hành trang thiết bị y tế :
5. Tên địa chỉ chủ sở hữu số lưu hành trang thiết bị y tế :
CÔNG TY TNHH TM VÀ DVKT NGUYỄN LÂM
107 Nguyễn Văn Công, Phường 03, Quận Gò Vấp, TP.HCM
6. Chủ sở hữu sản phẩm : Suzhou Yudu Medical Co.,ltd
7. Nhà sản xuất : Suzhou Yudu Medical Co.,ltd
8. Xuất xứ : CHINA
9. Ngày sản xuất hoặc hạn dùng : xem trên bao bì
10. Số lô hoặc số seri : xem tại mục LOT trên bao bì sản phẩm
11. Hướng dẫn tra cứu thông tin : xem trên tờ HDSD đi kèm hoặc tra cứu tại : www.nguyenlam.vn

Công ty công bố theo họ trang thiết bị y tế. công ty nộp một mẫu nhãn đại diện

Các thông tin có thể thay đổi trên nhãn theo họ chủng loại TBYT : Chủng loại thiết bị y tế: 500ml, 1000ml, 1200ml



Người đại diện hợp pháp của cơ sở
Ký tên (Ghi họ tên đầy đủ, chức danh)
Xác nhận bằng dấu hoặc chữ ký số



NGUYỄN CÁT TRIỀU