

DIRUI®

BCC-3D

Diluent

Instruction

【Name】 BCC-3D Diluent

【Specification】 20 L

【Intended Use】 Diluting sample

【Test Principle】 With appropriate osmotic pressure and conductivity, maintain the integrity of the original volume of blood cells in a certain period of time to guarantee access to the pulse size corresponding to cell volume.

【Ingredient】 Sodium sulfate, potassium chloride, sodium dihydrogen phosphate, dipotassium hydrogen phosphate.

【Storage Condition and Validity】 Stored at 2°C~30°C sealed and protected from sunlight, the validity is 24 months. The validity after opened is 60 days at 2°C~30°C.

【For Instrument】 DIRUI BCC-3600, BCC-3900, BCC-3960 automatic hematology analyzer.

【Sample Requirement】 EDTA should be used for anticoagulant (EDTA-2K, EDTA-3K or EDTA-2Na).

【Usage】 See the analyzer user manual.

【Important Notice】

1. For in vitro diagnosis.
2. Diluent should be stored at 2°C~30°C, can not be frozen.
3. Impurity should be avoided when it is being used.
4. Avoid contact with skin, clothes and eyes. Wash with plenty of water if it contacts the skin or clothes. Wash with plenty of water if it splashed into eyes and seeking medical care is recommended. If accidental swallow happened, call a physician immediately.
5. Sample, waste have potential infection, the operator should comply with the lab regulations. And dispose the waste according to the local regulations about medical waste, infectious waste and industrial waste.

【Reference】 Lin Faquan, Huang Mincheng, Huang Junqiang, Mo Wuning. SYSMEX SF-3000 Hematology Analyzer Diluent R&D and Application [J]. Guangxi Medicine, 2006(06): 1186-1187

【Symbol Explanation】

Please read the User Manual	Stored at	Manufacturer	Authorised Representative
In vitro diagnostic reagent	Lot Number	Valid until	Protected from sunlight
European in vitro Diagnostic Medical Device Directive 98/79/EC (IVDD)		Catalogue number	

【Instruction Approved and Modified Date】 02/2019



DIRUI INDUSTRIAL CO., LTD.

95 Yunhe Street, New & High Tech.
Development Zone Changchun, Jilin 130012 P.R.China
Tel: +86 431 85100409 Fax: +86 431 85172581
E-mail: dirui@dirui.com.cn http://www.dirui.com.cn

CE IVD

EMERGO EUROPE Prinsessegracht 20
2514 AP The Hague The Netherlands



GIÁM ĐỐC
Nguyễn Văn Luận