



Germany Technology

ENTERAL FEEDING BAG SET

INSTRUCTIONS FOR USE:

1. Close control clamp completely.
2. Fill bag with desired amount of formula.
3. Close and hang bag.
4. Add ice to ice pouch, if desired.
5. Insert drip chamber into drip chamber guides.
6. Open Clamp. Fill entire line with fluid. Close clamp.
7. Grasp silicon tubing and gently stretch around rollers. Insert retainer into retainer lock.
8. Open control clamp. Proceed with feeding

CAUTION:

- This set is intended for enteral feeding only
- Not for IV use.
- Discard after use.
- For single patient use only.
- It is recommended this device be replaced every 24 hours.



GIÁM ĐỐC

Nguyễn Cải Triều



Suzhou Yudu Medical Co., Ltd

Address: Eastern Xinzhuang (Zhangqiao) Town, 215552 Changshu, China