SPRAY NASAL DECONGESTIONNANT DECONGESTANT NASAL SPRAY

DESCRIPTION / DESCRIPTION

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The « Decongestant nasal spray » naturally relieves nasal congestion due to allergic rhinitis. Its formula made from natural hypertonic seawater acts by osmotic effect (suction effect) to mechanically decongest the nasal mucosa. By combining essential oils and natural hypertonic seawater, it enables to reduce the swelling of the nasal mucosa. By cleaning, it quickly enables the removing from nasal cavities of the mucus, which entraps airborne allergens such as pollen, acarids, animal hair or polluting particles, and at the same time it produces a sensation of freshness and released nasal breathing.

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Dispositif médical de classe I. / *Medical device class I.*

REGLE DE CLASSIFICATION SELON LA DIRECTIVE 93/42/CE / CLASSIFICATION RULE ACCORDING THE DIRECTIVE 93/42/EC

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Page 2 / 4

All invasive devices with respect to body orifices, other than surgically invasive devices and which are not intended for connection to an active medical device or which are intended for connection to an active medical device in Class I : are in Class IIa if they are intended for short-term use, except if they are used in the oral cavity as far as the pharynx, in an ear canal up to the ear drum or in a nasal cavity, in which case they are in Class I.

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Hypertonic Seawater with eucalyptus, mint, romane chamomile essential oils and propolis. Preservative-free.

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* *For nasal decongestion, in case of allergic rhinitis.*

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Children (≥ 3 years) and adults: 1 to 2 sprays per nostril, 3 times daily, depending on the degree of obstruction.

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- * *Remove the protection cap.*
- * *Gently insert the tip into your nostril, keeping it upright. Spray 1-2 times per nostril by pressing the pump.*
- * *Repeat with your other nostril.*
- * *Wipe off the surplus. Leave it and blow your nose if necessary.*
- * *Repeat 3 times daily, depending on the degree of obstruction.*

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- * *Ask your doctor or pharmacist for advice for use in subjects with an illness causing narrowing of the airways (e.g. asthma).*
- * *Do not use in case of hypersensitivity to seawater and/or a known allergy to one of the ingredients. Risk of allergy and/or sensitization to one of the ingredients is possible.*
- * *Do not give to children under the age of 36 months.*
- * *Do not use in children with a history of febrile seizures.*

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Sensations of tingling or temporary irritation may appear at the start of use.

CONDITIONS DE CONSERVATION / STORAGE CONDITIONS

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- * *Bottle to be used within 3 months after opening.*
- * *Do not use after the limit date of use specified on the carton.*
- * *Store below 30°C.*
- * *KEEP OUT OF THE REACH AND SIGHT OF CHILDREN.*

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Before opening: 24 months from date of manufacture.
After opening: 3 months.

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