



Germany Technology

# CAST PADDING

## INSTRUCTIONS FOR USE:

[Intended use] For fracture fixation.

[Usage method] According to the size of the injured area, choose a suitable size plaster bandage

Immersion: Use commonly used warm water to soak the plaster bandage in water obliquely for 5-10 seconds.

Until the bubbles disappear.

Squeeze dry: take the bandage out of the water and squeeze it from both ends to the center to drain the excess water

Bandaging: The bandage should be wrapped around and overlaid. Don't overtighten the bandage. Wrap the general part.

Around 6-8 layers, 8-10 layers for important parts.

Leveling: Leveling is performed at the same time of bandaging, remove air bubbles in the bandage and make the bonding between layers evenly.

And modify the appearance to achieve a smooth and even appearance; do not touch the plaster when it starts to set.

### 【Precautions】

All plaster bandages should be sealed and packaged, and special attention should be paid to avoiding damage to the package and causing external voids during storage.

Air enters the bandage and becomes damp.

### 【Storage, Transportation and Storage Conditions】

After packaging, the product should be stored in a clean room with a relative humidity not exceeding 75%, a temperature not exceeding 50 °C, no corrosive gas, and well-ventilated.



GIÁM ĐỐC

*Nguyễn Cải Triệu*



Suzhou Yudu Medical Co., Ltd

Address: Eastern Xinzhuang (Zhangqiao) Town, 21552 Changshu, China