

HƯỚNG DẪN SỬ DỤNG



Tên trang thiết bị y tế	Băng keo cuộn
Chủng loại	Nichipore
Chỉ định	Cố định băng, gạc phẫu thuật cotton, băng ép Cố định kim tiêm và kim lườn Cố định ống thông và ống
Hướng dẫn sử dụng	Làm sạch và khô vết thương trước khi dán băng.
Chú ý khi dùng	Không băng trực tiếp lên vết thương. Nếu phát hiện phát ban, hoặc ngứa... trong khi sử dụng sản phẩm, xin vui lòng ngưng sử dụng. Tháo băng: bóc chậm dọc theo chiều lông trên cơ thể để ngăn ngừa tổn thương da.
Bảo quản	Giữ ở nhiệt độ phòng, tránh nước, nhiệt độ cao, độ ẩm cao và ánh sáng mặt trời trực tiếp.



USER MANUAL FOR NICHIPORE



Overview

This tape consists of pulp-polyester nonwoven fabric and acrylic adhesive.

Features

Highly versatile

- Because it fits well on the skin, it has excellent fixation on areas with lots of sebum such as forehead. This tape can be used for many purposes.

Easy to hand-tearing

- Easy to treat because it can be easily torn by hand.

X-ray transmission

- There is no shadow of the tape during X-ray photography.

Reduces UV transmission

- It can cut off UV rays, one of the causes of pigmentation.

Moderate moisture permeability and air permeability

Applications

- For fixing gauze and bandages
- For fixing catheters and tubes
- For scar protection

Precautions for Use

- Do not apply directly to a wound.
- If symptoms such as rash, redness and itchiness appear during use, discontinue use.



- Please remove slowly along the flow of hair so as not to damage the skin.

Precautions for Storage

- Store at room temperature, away from water, high temperatures, high humidity and direct sunlight.

