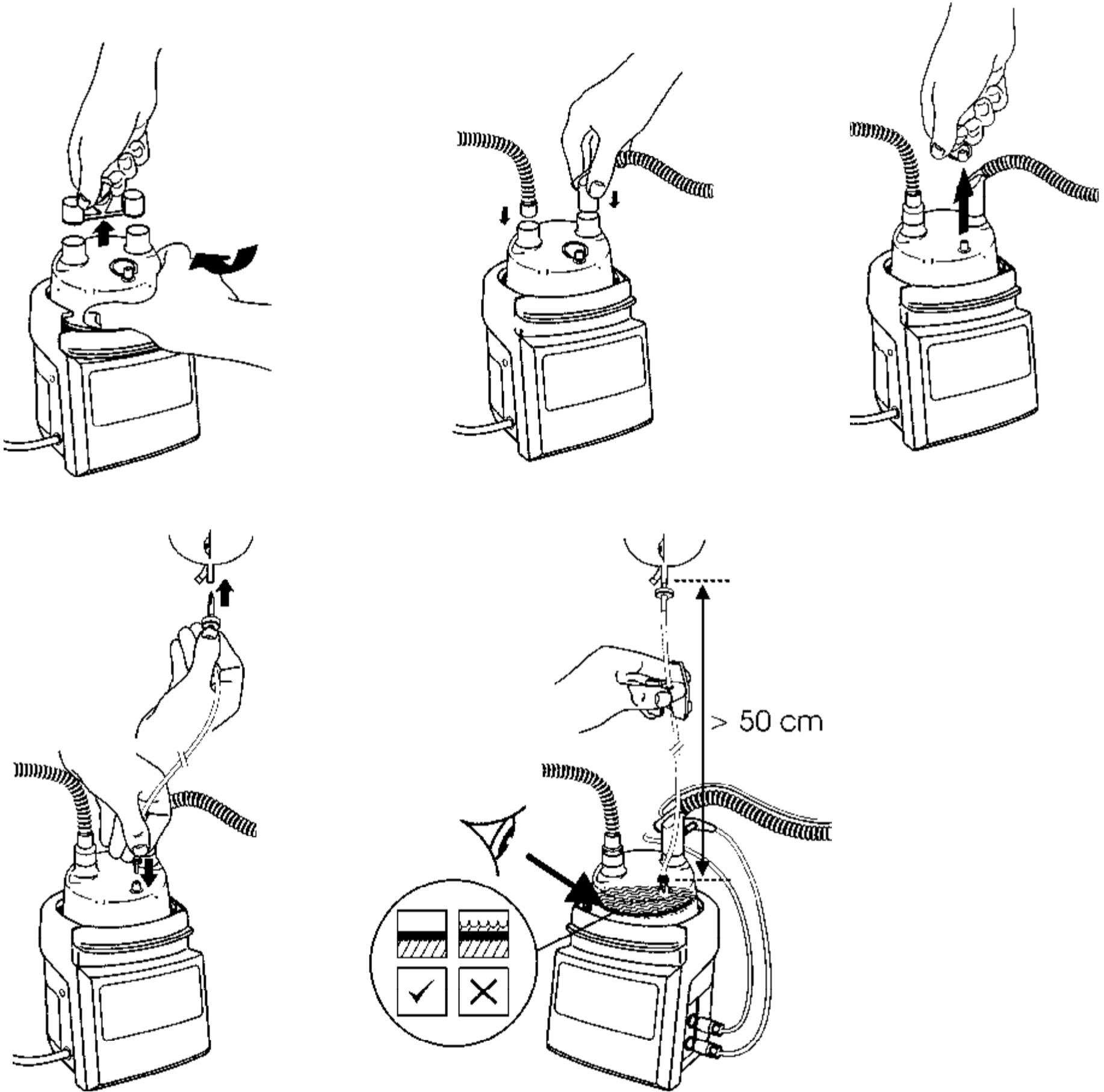
CÔNG TY TNHH ASIA ACTUAL VIETNAM
MST: 0316765196

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English ^(en)

HUMIDIFICATION CHAMBER

The chamber is designed for humidifying gas with Fisher & Paykel Healthcare breathing circuits.

TECHNICAL SPECIFICATIONS

Compatible with Fisher & Paykel Healthcare MR850 humidifiers. Refer to humidifier user instructions.

INTERFACE CONNECTIONS	ISO 5356-1 Conical Connectors
------------------------------	-------------------------------

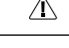
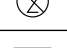
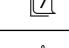
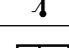
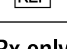
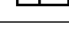
MR210 GAS PORTS	
Inlet:	22 mm Female
Outlet:	22 mm Male

MR250 GAS PORTS	
Inlet:	22 mm Male
Outlet:	22 mm Male

MAXIMUM WATER CAPACITY COMPRESSIBLE VOLUME	
Full:	220 mL
Empty:	480 mL

COMPLIANCE	
Full:	0.3 mL/cmH ₂ O
Empty:	0.5 mL/cmH ₂ O
MAXIMUM OPERATING PRESSURE	20 kPa
MAXIMUM PEAK FLOW	200 L/min
GAS LEAKAGE @ 60 cmH₂O	<10 mL/min

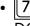
SYMBOL DEFINITIONS

	Water level too high		CE Marking 93/42/EEC
	Maximum water level		Date of manufacture
	Consult operating instructions		Manufacturer
	Not made with phthalates		Use-by date
	Not made with natural rubber latex		Caution/Consult instructions for use
	Single use		7 Days maximum use
	Lot number		Transportation and storage temperature limits
	Reference number		European Union authorised representative
Rx only	Prescription only		

WARNINGS, CAUTIONS, AND NOTES

WARNINGS

- DO NOT reuse this product. Reuse may result in transmission of infectious substances, interruption to treatment, serious harm or death.
- The use of breathing circuits, chambers, accessories, or combinations which are not approved by Fisher & Paykel Healthcare may result in poor humidification system performance, ventilator malfunction, and harm to the patient/ user.
- DO NOT touch the heater-plate or chamber base. Surfaces may exceed 85 °C. Failure to comply may result in a skin burn.

- Failure to comply with the following warnings may impair performance of the device or compromise safety (including potentially cause serious harm):**
- When mounting a humidifier adjacent to a patient, ensure that the humidifier is always positioned lower than the patient.
-  DO NOT use beyond 7 days maximum duration of use.
- DO NOT soak, wash, or sterilize this product. Avoid contact with chemicals, cleaning agents, or hand sanitizers.
- DO NOT use the chamber if the seals are not intact when received.
- Ensure the humidifier is not tilted. Tilting the humidifier may result in water entering the breathing circuit.
- Use USP sterile water or equivalent for humidification. DO NOT add other substances to the water.
- Check all connections are tight before use.
- Ensure that the water level in the chamber is periodically monitored. Refill when necessary.
- DO NOT fill the chamber with water in excess of 37 °C.
- Ensure appropriate ventilator or flow source alarms are set before connecting breathing set to patient.
- DO NOT fill the chamber above the maximum fill level line. Liquid could enter the breathing circuit if the chamber is overfilled.
- If disconnecting the breathing circuit from the chamber, the humidifier MUST be put into stand-by or turned off to prevent excessive temperatures.
- Remove water feed inlet plug only prior to inserting water feed set. Never reinsert water feed inlet plug as this may cause leakage.
- Ensure the water bag is placed below the level of the humidifier after the chamber has been filled.
- In the event of the chamber leaking, switch the humidifier off and replace the chamber.
- California residents please be advised of the following, pursuant

to Proposition 65: This product contains chemicals known to the State of California to cause cancer, birth defects and other reproductive harm. For more information, please visit: www.fphcare.com/prop65

NOTES

- For use under the supervision of trained medical personnel.
- Refer to hospital protocol for disposal methods. User may be exposed to breathing tract fluids during disposal.
- The responsible organization is accountable for the compatibility of the humidifier and all of the parts and accessories used to connect to the patient and other equipment before use.

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