

Usage Method and Notice

4.1 the usage method of surgical blade

- a. Disposable surgical blades must be used by professional nurses.
- b. Choose a meet size, according to the patient's requirement for surgery.
- c. Dilacerate the packing , take out the small package and tear open the plastic bag. Remove the surgical blade and use it.

4.2 Notice:

- a. Disposable surgical blades can be used after opening the seal and used only once, to prevent cross infection.
- b Don't use, if the pack is broken
- c. Expiration date of disposable surgical blades is 5 years, be forbidden to use when it expires.
- D.Disposable surgical blade's storage temperature $\leq 40^{\circ}\text{C}$, humidity $\leq 80\%$, no corrosive gas and ventilation in good indoor.

5.0 Expiration Date of Product

Expiration date of disposable surgical blades is 5 years