



5F, NO. 63, Jingke Central Rd., Nantun Dist., Taichung City 408018, Taiwan

☎ 886 4 2359 4147 ☎ 886 4 2359 9338 ✉ service@tronjen-med.com
🌐 www.tronjen-med.com

INSTRUCTION FOR USE

1. SOGIBAND

Clean the wounds and make it dry thoroughly.
Apply plaster and press down the edges firmly.
If irritation occurs or bleeding continues, stop use and seek medical advice.

2. SOGICELL

Gently clean the wound then dry.
Position the dressing centrally over the wound and press down the edges firmly.
Replace with a fresh dressing when the dressing loosens or as necessary.

3. SOGIDRESSING

Gently clean the wound.
Apply the dressing to the wound.
Apply a fresh dressing as required.

4. GOGIFLEX:

Gently clean the wound.
Apply the dressing to the wound.
Apply a PU film onto top to fix dressing in place .
Apply a fresh film as required.
OR
Peel off the shorter side of release paper first.
Holding two wings and put them toward the direction of catheter then the side of PU film is right on the top of needle area.
Peel off the release paper on the other and make the wings fix the catheter in place.
Remove the carrier on the top from the midline of the blue bar.
Smooth dressing edges to increase adhesion .

5. SOGIPLASTER

For optimal adherence, clean skin and pat dry.
Smooth tape on skin with gentle finger pressure.

6. SOGIROLL

For optimal adherence, clean skin and pat dry.
Smooth tape on skin with gentle finger pressure.

7. SOGISYVAL

For optimal adherence, clean skin and pat dry.
Smooth tape on skin with gentle finger pressure.

Prepared on July 26th, 2022.