		Ningbo Greetmed Medical Instruments Co., Ltd.				Prepared by	Dong Jingjing
		Instruction for Use				Checked by	Chen Xiaoping
						Approved	Li Guirong
Edition	A/0	Doc. No	GM/CE-I-19-Annex 05	Effective Date	2021-06-07	Total page	Page 1 of 3



Instruction for Use

【Product Name】

Disposable Plastic Forceps

【Model/Szie】

Size: 10.8cm, 14cm, 11.5cm, 12.5cm, 18cm

【Intended use】

Disposable Plastic Forceps is suitable for use in outpatient clinics or operations to clamp dressings or utensils for cleaning or disinfection.

【Product picture】



【Cautions and Warnings】

- 1) The product is for single use, do not reuse.
- 2) If package of product is damaged, dot not use
- 3) If any adverse event occurs, it should be reported to the manufacturer and competent authority of the Member State where the user and/or patient is located.

【Disposal of waste】

Products after use should be destroyed and disposed of according to local/national regulations. It is strictly forbidden to discard it randomly.

【Directions】

1. Please check if the package is damaged before use.
2. Take out the Disposable Plastic Forceps from package.
3. After use, please dispose of the waste in accordance with the requirements of the hospital or environmental protection department.

【Expiry】


Valid period of product is 5 years.

【Production Date】

Refer to the package label.

【Storage and transport conditions】

- 1) Keep away from moisture, rain, light and heat during transportation.













		Ningbo Greetmed Medical Instruments Co., Ltd.				Prepared by	Dong Jingjing
		Instruction for Use				Checked by	Chen Xiaoping
						Approved	Li Guirong
Edition	A/0	Doc. No	GM/CE-I-19-Annex 05	Effective Date	2021-06-07	Total page	Page 2 of 3

2) Store in a well ventilated, dry, cool, far away from heat source and without corrosive gas.

【Sterilization】

Non-sterile

【Symbol Description】

	CE Mark: conforms to essential requirements of the Medical Device Regulation 2017/745 (EU)
	Indicates the need for the user to consult the instructions for use
	Date of manufacture, Indicates the date when the medical device was manufactured
	Manufacturer, Indicates the medical device manufacturer, as defined in Medical Device Regulation 2017/745 (EU)
	Use-by date, Indicates the date after which the medical device is not to be used
	Do not re-use, indicates a medical device that is intended for one use, or for use on a single patient during a single procedure
	Non-sterile
	Do not use if package is damaged, indicates a medical device that should not be used if the package has been damaged or opened
UDI-DI	UDI-DI code of product identification
	Batch code, Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Keep away from sunlight, indicates a medical device that needs protection from light sources.
	Keep dry, indicates a medical device that needs to be protected from moisture.
	Medical Device mark


【Manufacturer】

Name: Ningbo Greetmed Medical Instruments Co., Ltd.

Address: 16F-1, Building 1, No.98 Chuangyuan Road, Hi-Tech Zone, 315042 Ningbo, Zhejiang Province, PEOPLE'S REPUBLIC OF CHINA

Tel.: 86-574-87739070

Fax: 86-574-87722360

		Ningbo Greetmed Medical Instruments Co., Ltd.					
		Instruction for Use				Prepared by	Dong Jingjing
						Checked by	Chen Xiaoping
						Approved	Li Guirong
Edition	A/0	Doc. No	GM/CE-I-19-Annex 05	Effective Date	2021-06-07	Total page	Page 3 of 3

【EU Representative】

Name: MedPath GmbH

Address: Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

Tel.: +49(0)89 189174474

Fax: +49(0)89 5485 8884

E-mail: info@medpath.pro