

HƯỚNG DẪN SỬ DỤNG TIẾNG ANH

Tài liệu được xác nhận bằng chữ ký số

Hà Nội, ngày 21 tháng 07 năm 2022

Người đại diện hợp pháp của cơ sở

GIÁM ĐỐC

Nguyễn Thị Kim Chi

Premier Resolution Mobile Phase 2 Reagent

REF 01-03-0083

Fill 940mL

REF 01-03-0085

Fill 3.8L

OTHER LANGUAGES

Pour d'autres langues
Für andere Sprachen
Para otras lenguas
Per le altre lingue
Dla innych języków

Para outras línguas
Για τις άλλες
λώσσες
Für andra språk
For andre språk



www.trinitybiotech.com

INTENDED USE

The Trinity Biotech Premier Resolution Mobile Phase 2 Reagent is intended for use with Premier Resolution analyzer. No substitutions are permitted, registered, cleared or authorized. No other uses are intended, registered, cleared or authorized.

The Trinity Biotech Premier Resolution analyzer is an automated High Performance Liquid Chromatography (HPLC) system which performs the separation of hemoglobin fractions in human blood samples for the quantitative analysis of normal hemoglobin (A, A₂, and F), and the qualitative detection by Quick Scan (QS) assay and semi-quantitative analysis by High Resolution (HR) assay of major variant hemoglobin S, C, D-Los Angeles, and E in adult, adolescent, children and infant populations. The assays are performed on human blood samples collected in tubes containing K₂EDTA, K₃EDTA, Sodium Heparin or Lithium Heparin as anticoagulant.

The Trinity Biotech Premier Resolution analyzer is intended for professional laboratory use only.

The Trinity Biotech Premier Resolution analyzer is intended for use with analytical components and reagents provided by Trinity Biotech.

For *in vitro* diagnostic use.

SUMMARY AND EXPLANATION OF TEST

Premier Resolution Mobile Phase 2 is a reagent used on the Premier Resolution HPLC analyzer in the identification of altered hemoglobin species.

Reagents are performance validated to ensure accuracy and precision when used with the Trinity Biotech Premier Resolution analyzer.

Reagent is ready for use.

STORAGE AND STABILITY



Store at ambient temperatures (2 – 28°C). Do not allow to freeze.



The reagent is stable until the expiration date indicated on the label when kept tightly closed. See reagent label for the expiration date. Do not use after the expiration date.

PRECAUTIONS

For *in vitro* diagnostic use only. Avoid skin contact. Consult the product MSDS for safety information. This reagent is used in conjunction with blood testing equipment and warrants handling under universal precaution procedures for safety.

ORDERING INFORMATION

Reference No.	Item Description	Fill Volume
01-03-0083	Premier Resolution Mobile Phase 2 Reagent	940mL
01-03-0085	Premier Resolution Mobile Phase 2 Reagent	3.8L

CONTACT INFORMATION



Trinity Biotech
Kansas City, MO 64132 USA
Tel. 1 800-325-3424
Fax: 1 816-361-1974



Trinity Biotech plc
Bray Co. Wicklow, Ireland
Tel. 353 1 2769800
Fax 353 1 2769888
www.trinitybiotech.com

REGULATORY

