

β-HCG Test Kit User Manual

(Dry Fluorescence Immunoassay)

[PRODUCT NAME]

β-HCG Test Kit (Dry Fluorescence Immunoassay)

[PACKAGE SPECIFICATION]

- 1 test/kit
- 5 tests/kit
- 25 tests/kit
- 50 tests/kit
- 100 tests/kit

[INTENDED USE]

β-HCG Test Kit (Dry Fluorescence Immunoassay) is intended for in-vitro quantitative measurement of β-HCG in human serum and plasma. This test is used as an important aid in pregnancy test.

[TEST PRINCIPLE]

β-HCG Test Kit (Dry Fluorescence Immunoassay) uses the principle of antigen-antibody reaction. The testing sample will migrate forward due to capillary action, then the β-HCG of the sample will combine with antibody which is attached to fluorescence microspheres. This marked complex is attached to the detection area of immobilized antibody, and the other fluorescence microspheres are attached to the control area. When the test strip is inserted into the analyzer, the analyzer automatically scans two ribbons and detected the fluorescence intensity of the composite emission from the testing area and the control area. The ratio of the two fluorescence values was used to calculate the content of the detected substances.

[MAIN COMPONENTS]

1. β-HCG test strip in a sealed pouch with desiccant.....25 tests
2. Sample diluent.....25 pieces
3. QR code card for calibration.....1 piece
4. User Manual.....1 piece
5. Quantitative suction and dropping tube (Optional).

Note: Do not mix or interchange different batches of kit.

[STORAGE AND VALIDITY]

Store the test kit at 4°C-30°C, with a valid period of 18 months.
 Test strip should be used within 30 minutes once the foil pouch is opened.

[APPLICABLE DEVICES]

1. FIC-S100 Dry Fluorescence Immunoassay Analyzer
2. LS-1000 Dry Fluorescence Immunoassay Analyzer
3. LS-2000 Dry Fluorescence Immunoassay Analyzer
4. LS-1100 Dry Fluorescence Immunoassay Analyzer
5. LS-2100 Dry Fluorescence Immunoassay Analyzer
6. LS-4000 Hand-Held Dry Fluorescence Immunoassay Analyzer
7. LS-7000 Microfluidic and Dry Fluorescence Immunoassay Analyzer

[SAMPLE REQUIREMENT]

1. Used for human **serum and plasma**. Other bodily fluids and samples may not get the accurate result.
2. Plasma sample can be anticoagulant with heparin and sodium citrate under aseptic conditions.
3. At room temperature, the test should be performed within 4 hours after the sample collection.
4. Serum or plasma sample can be stored at 2°C-8°C for 3 days at most. If more than 3 days, it should be stored at -20°C. It is suggested to use fresh sample to test. Stale blood sample may not get accurate result.
5. The sample before testing should be recovered to room temperature (15°C-30°C).
6. **Sample Volume: 50μL**

[TEST PROCEDURE]

1. Collect samples according to user manual.
2. Before the test, the sample, test strip and diluent should be recovered to room temperature (15°C-30°C).

For LS-1100:

3. Perform QR code calibration when necessary. (Details refer to LS-1100 User Manual).
4. On the main interface of LS-1100, press “Test” icon to enter testing interface. Input patient information, sample information, doctor information when necessary. (Details refer to LS-1100 User Manual)
5. Remove test strip from sealed pouch and put it on a clean table, horizontally placed.
6. Using pipette to deliver 50μL of sample into one tube of sample diluent. Mix gently and thoroughly. And then drop 100μL of mixed fluid into the sample port in the test strip.

7. Reaction Time: 15 minutes

For panel inside: Insert the test strip into the analyzer immediately after sample dispensing. Then click “Test”.

For panel outside: After reaction time 15 minutes is elapsed, insert the test strip into the analyzer and then click “Test”.

8. The result will be shown on the screen and printed automatically.

Notes: It is required to perform QR code calibration when starting to use one new batch of kit.

[EXPECTED VALUE]

Cut-Off Value: 5.0mIU/mL

The cut-off value for β-HCG was determined by testing samples from 200 apparently healthy individuals. The 95th percentile of the concentration for β-HCG is 5.0mIU/mL.

It is recommended that each laboratory establish its own reference range for the population it serves.

[INTERPRETATION OF RESULT]

1. If the test result of the sample is more than 20000mIU/mL, the analyzer displays “>20000mIU/mL”, and if the result is less than 2.0mIU/mL, the analyzer displays “<2.0mIU/mL”. Specific data can be exported through related software as needed.

- When the sample concentration exceeds the detection limit, the maximum dilution ratio is 3 times when the sample is diluted with calf serum or negative sample.

[LIMITATION]

- This kit is only for serum and plasma test.
- The test result of this kit is only one of the diagnostic aids for the clinicians.
- The result of the test should be evaluated in the context of all the clinical and laboratory data available. In those instances where the laboratory results do not agree with the clinical evaluation, additional tests should be performed accordingly.

[PRODUCT PERFORMANCE]

- Measuring Range: 2.0-20000mIU/mL.
- Lower Detection Limit: ≤ 2.0 mIU/mL.
- Accuracy: Verify with comparison experiments, the relative deviation $\leq 15\%$, the correlation coefficient $r \geq 0.990$.
- Within-Run Precision: $\leq 15\%$.
- Between-Run Precision: $\leq 15\%$.
- Hook Test: No hook effect with high concentration sample. Hook test was conducted with reference material exceeding the upper limit of linear range, and the detection result was greater than the upper limit of detection.

[PRECAUTIONS]

- Only used for in vitro diagnostics.
- Do not use the kit beyond the expiration date.
- After the test strip is removed from the sealed pouch, it should be tested as soon as possible to avoid excessive time in the air, resulting in dampness.
- Do not reuse the test strip.
- The damaged test strip or package cannot be used.
- Do not mix the components of different kits.

[REFERENCES]

- Kuang qingmei, zhang wenhui, Chen wenyong, li yinfang. The value of serum human chorionic gonadotropin and serum lipid in early pregnancy in predicting pregnancy hypertension[J]. Clinical medicine engineering, 2015, 22(10): 1349-1350+1353.
- Xue wj, wang w, wang xl, huang hj. Experimental evaluation of determination of human chorionic gonadotropin beta in urine by Access chemiluminescence[J]. Shanghai Journal of Medical Laboratory Science, 2003(02): 105-106. (in Chinese with English abstract)
- Xu LAN, Chen zhuqin. Diagnostic value of human chorionic gonadotropin measurement of betaemia in abdominal cavity for ectopic pregnancy[J]. Journal of the Third Military Medical University, 1994(02): 155.
- Li w, Du wj. Value of serum beta human chorionic gonadotropin, progesterone, oncoantigen 125 and endometrial thickness in the diagnosis of early ectopic pregnancy [J]. Chinese Journal of Obstetrics and Gynecology, 2010, 26(10): 759-762.



Lansion Biotechnology Co., Ltd.
 Add: No.2, Qiande Road, Jiangning District, Nanjing, China
 Tel: 86-25-58577600
 Fax: 86-25-58758600
 E-mail: biz@lansionbio.com
 Website: en.lansionbio.com



Llins Service & Consulting GmbH
 Address: Am Heiligenhaus 7,69126, Heidelberg, Germany
 Tel: +49 176 63866127

Revision Date: May 31, 2019

Version Number: 0.1

Production date and expiration see the label.



GIÁM ĐỐC
Tăng Đỗ Dũng